

BUSTER Quick Collar



BUSTER Quick Collar

The BUSTER Quick Collar has a touch fastener system for easy customised adaptation to each animal. The collar is padded with a soft outer silicone edge to avoid marking or scraping and a soft inner silicone edge for a comfortable fit. The collar is provided with neck holes for secure fastening to the dog's neck collar with ie. gauze or a strip.

- Quick custom fit for the individual animal
- Easy application and removal
- Full vision
- Full body coverage
- Made of soft silicone and 100% polypropylene
- 7 sizes and a Starter Pack available

Recommended use for BUSTER Quick Collar

- Postoperative care
- Wound care
- Spay/neuter recovery
- Hot spots and other skin ailments
- Skin diseases



Unique features and benefits

Safety and perfect fit

The collar is provided with neck holes for secure fastening to the dog's neck collar with ie. gauze or a strip.

Easy application

The collar has a touch fastener system for easy and customized adaptation to each animal.

Soft padded edges

The collar is padded with a soft outer silicone edge to avoid scratching humans and furniture and a soft inner silicone edge for optimal a comfortable fit.


Full body coverage

The collar offers a full body coverage and protection preventing the animal from reaching certain areas of the body.

Instruction for use


Step 1

- Fold the collar around the neck
- Mark the selected position
- Leave two fingers width between the collar and neck
- Keep the touch fastener mated throughout the sizing procedure.

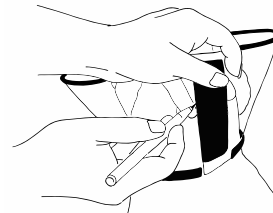


Step 2

- Remove the adhesive liner from the touch fastener.

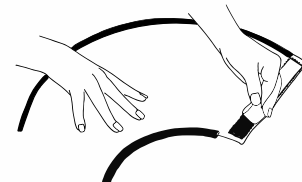


Step 3

- Apply the adhesive part to the marked position
- Cut off redundant collar material

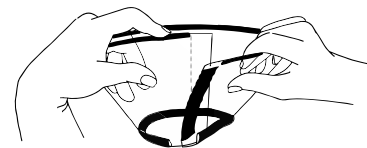

Step 4

- Place the collar on the pet's neck.

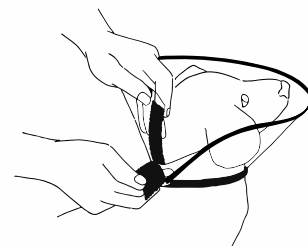

Instruction for use

Scan QR code to watch the video

Caution to Pet Owners

It is normal that pets are responding negatively the first time, they are introduced to the e-collar. To ensure your pet's safety while wearing the e-collar keep it indoors in a restricted area. If you are concerned about your pet's behavior or recovery contact your veterinarian.

Collar Maintenance:

Wipe with a damp cloth.



Cat. No	Size cm / in	Pk. Size	Suitable for	Kg	lb
273511	7.5 cm / 3"	5/pk	Pomeranian, Chihuahua	1-3	2-6
273512	10 cm / 4"	5/pk	Yorkshire terrier, Bichon Frise, Maltese	3-5	6-11
273513	12.5 cm / 5	5/pk	Danish/Swedish Farmdog, Papillon, Cairn Terrier, Border Terrier	5-10	11-22
273514	15 cm / 6"	5/pk	Dutch Kooikerhondje, Scottish Terrier, Beagle, Irish Terrier	10-15	22-33
273515	20 cm / 8"	5/pk	Australian Shepherd, English Springer Spaniel, Staffordshire Bull Terrier	15-25	33-55
273517	25 cm / 10"	5/pk	Labrador Retriever, Siberian Husky, Irish Setter	25-30	55-66
273518	30 cm / 12"	5/pk	Golden Retriever, Boxer, Collie, German Shepherd	25-35	55-77

273519 Starter Pack of size 7.5 - 30 cm 7/pk