

Urinary Concept



KRUUSE Urinary Concept

We know that our many customers across the continents expect KRUUSE to be at the forefront of innovation and development – and we are now, once again, able to introduce a **1st in veterinary medicine**. In line with our present range of products, this product is based on our belief that veterinary patients should have access to the same standard of consumables as human patients. We have therefore worked together with a leading manufacturer of high quality human urinary catheters to develop a **urinary catheter** for dogs.

What is so revolutionary about the urinary catheter? The full length of the catheter has a **hydrophilic coating**. When this coating is activated in water, the result is a very smooth and viscous surface. It should be noted that the full length of the catheter becomes smooth and viscous. Gel applied to the tip of the catheter will never achieve the same effect. The hydrophilic coating significantly decreases friction in the dog's urethra – providing obvious benefits to canine patients.

Hydrophilic coated catheters have been available in human medicine for decades, and they have been tested and used extensively in humans. These catheters are, not surprisingly, preferred by human patients, as the hydrophilic coating **reduces friction** in the urethra. This helps to decrease patient discomfort and to prevent iatrogenic damage to the urethra. Hydrophilic coated catheters provide additional benefits such as less microscopic haematuria and decreased incidence of UTI (urinary tract infection).

Now hydrophilic coated urinary catheters are **available for dogs**. After all, why should canine patients not have the same benefits as human patients when urinary catheterization is indicated? A dog feels discomfort when catheterized exactly the same way a human patient does.

You can read more about KRUUSE HC Urinary Catheter in this brochure where you also will find information about the other Urinary Concept products, including our full range of urinary catheters for dogs and cats.



BUSTER HC Urinary Catheter

-Hydrophilic Coated

**KRUUSE INTRODUCES A 1ST
IN VETERINARY MEDICINE**

In line with our present range of products, this product is based on our belief that veterinary patients should have access to the same standard of consumables as human patients.

What is so revolutionary about this urinary catheter? The full length of the catheter has a hydrophilic coating. When the coating is activated in water, the result is a very smooth and viscous surface throughout the entire length of the catheter. The hydrophilic coating significantly decreases friction in the dog's urethra – providing obvious benefits to canine patients.

This helps to decrease patient discomfort and to prevent iatrogenic damage to the urethra. Hydrophilic coated catheters provide additional benefits such as less microscopic haematuria and decreased incidence of iatrogenic UTI (urinary tract infection).

How to use...



Open the pouch



Pour suitable water in the pouch



Fix it on a tile if desired.
Wait 30 sec



The catheter is ready for catheterization

NB: The catheter must not be left in the urethra for more than 10 minutes



Scan the QR-code to see the video "How to.."



Assortment

Cat. No	Size, Metric	Size, Imperial	Colour	Package
273866	1,6 mm x 50 cm	5 Fr x 20 in		12/pk
273867	2,0 mm x 50 cm	6 Fr x 20 in		12/pk
273868	2,6 mm x 50 cm	8 Fr x 20 in		12/pk
273869	3,3 mm x 50 cm	10 Fr x 20 in		12/pk



Features and benefits

Viscous surface

Hydrophilic coating results in a very smooth and viscous surface

Reduce friction

Hydrophilic coating reduces friction of the catheter in the urethra and helps prevent iatrogenic damage of urethra

Lower infection risk

Hydrophilic coating and atraumatic drainage holes significantly reduce the risk of bladder infection

Uncomplicated catheterization

No need for using gel on the catheter - this simplifies the management with catheters

Even distribution

The coating is more even applied on the catheter compared to gel

Easy size recognition

Colour-coded luer lock fitting and label for easy size recognition

Convenient and fast access to the catheters

User-friendly end-opening box and individually packed pouches



BUSTER NC Urinary Catheter

-Non Coated

BUSTER NC Urinary Catheter is a soft and biocompatible catheter. It is made from DEHP- free (phthalate free) medical grade PVC. The soft catheter is less likely to cause damages to the mucous membranes in urethra and in the bladder compared to the more rigid catheters.

- Atraumatic polished lateral drainage holes
- High biocompatibility
- Suitable for short and medium-long placement (up to 3 days)
- Colour-coded luer lock fitting for easy size recognition
- User-friendly end-opening box
- 12/pk

FOR INCREASED PATIENT COMFORT

How to use...



Open the pouch



Add gel



The catheter is ready for catheterization

Assortment

Cat. No	Size, Metric	Size, Imperial	Colour
273875	1,6 mm x 50 cm	5 Fr x 20 in	Grey
273876	2,0 mm x 50 cm	6 Fr x 20 in	Green
273877	2,6 mm x 50 cm	8 Fr x 20 in	Blue
273878	3,3 mm x 50 cm	10 Fr x 20 in	Black



BUSTER Dog Catheter


The popular range of BUSTER Dog Catheter now consists of 7 sizes of catheters!

- Female luer lock fitting
- Rigid though flexible polypropylene catheter tubing for easy introduction
- Rounded end and atraumatic side holes to avoid tissue damage
- Userfriendly blister packaging
- 12/pk

Cat. No	Size, Metric	Size, Imperial
273405	1.3 mm x 50 cm	4 Fr x 20 in
273406	1.6 mm x 50 cm	5 Fr x 20 in
273410	2.0 mm x 50 cm	6 Fr x 20 in
273411	2.3 mm x 50 cm	7 Fr x 20 in
273420	2.6 mm x 50 cm	8 Fr x 20 in
273421	3.0 mm x 50 cm	9 Fr x 20 in
273430	3.3 mm x 50 cm	10Fr x 20 in



Rounded end and atraumatic side holes

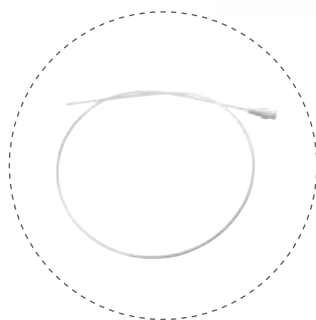

Flexible polypropylene tubing



Female luer lock fitting




BUSTER Foley Catheter, Silicone

2-way catheter in 100% silicone with 3-5 cc (ml) retention balloon and stylet.
Catheter has rounded tip and atraumatic side holes, and as silicone is very biocompatible, it is suitable for longer placement in urethra/bladder (up to 7 days).


- Silicone catheter with high biocompatibility
- Suitable for long term placement
- Easy-removable steel wire
- Colour-coded cap for easy size recognition
- 5/pk




How to use...

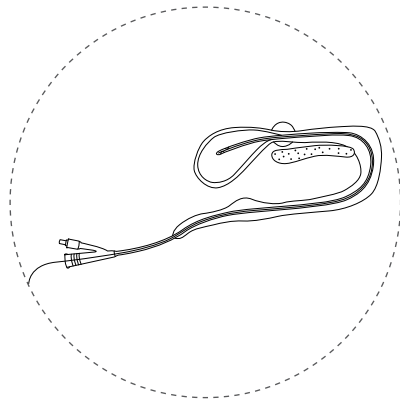

Insert the catheter through urethra into the bladder

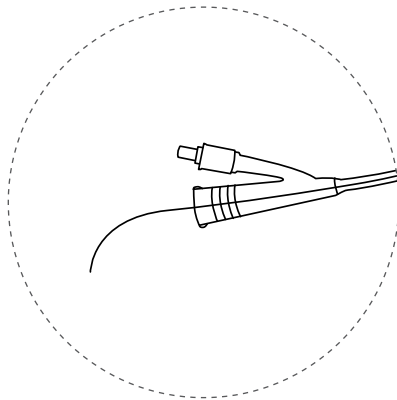

Gently remove the steel wire

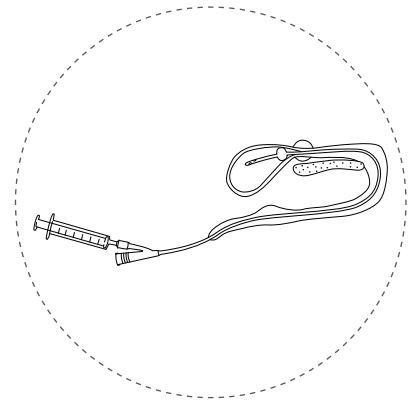

Fill the balloon with isotonic saline

Assortment

Cat. No	Size, Metric	Size, Imperial	Colour
273860	2 mm x 30 cm	6 Fr x 12 in	●
273861	2 mm x 55 cm	6 Fr x 22 in	●
273862	2.6 mm x 30 cm	8 Fr x 12 in	●
273863	2.6 mm x 55 cm	8 Fr x 22 in	●
273858	2.6 mm x 70 cm	8 Fr x 28 in	●
273864	3.3 mm x 30 cm	10 Fr x 12 in	●
273865	3.3 mm x 55 cm	10 Fr x 22 in	●
273859	3.3 mm x 70 cm	10 Fr x 28 in	●




Period of Placement in Situ	Instant (< 10 minutes)	Short term (0-1 day)	Medium term (1-3 days)	Long term (0-7 days)																																																																																	
Clinical Applications	<ul style="list-style-type: none"> ■ Urine sample collection ■ Check of passage in urethra ■ Bladder tension relief ■ Bladder lavage 	<ul style="list-style-type: none"> ■ Urine sample collection ■ Check of passage in urethra ■ Bladder tension relief ■ Bladder lavage ■ Catheterization for drainage purposes during surgery or hospitalization 	<ul style="list-style-type: none"> ■ Urine sample collection ■ Check of passage in urethra ■ Bladder tension relief ■ Bladder lavage ■ Catheterization for drainage purposes during surgery or hospitalization 	<ul style="list-style-type: none"> ■ Urine sample collection ■ Bladder tension relief ■ Bladder lavage ■ Catheterization for drainage purposes during surgery or hospitalization 																																																																																	
Recommended Catheters	<p>BUSTER HC Urinary Catheter</p> <ul style="list-style-type: none"> ■ No need for gel ■ Soft and comfortable ■ Colour-coded hub and label for easy size recognition ■ Luer lock on hub allows easy connection to syringe 	<p>BUSTER Male Dog Catheter</p> <ul style="list-style-type: none"> ■ More rigid catheter ■ Luer lock on hub allows easy connection to syringe or urine bag 	<p>BUSTER NC Urinary Catheter</p> <ul style="list-style-type: none"> ■ Soft and comfortable ■ Colour-coded hub and label for easy size recognition ■ Luer lock on hub allows easy connection to syringe or urine bag 	<p>BUSTER Foley Catheter</p> <ul style="list-style-type: none"> ■ Very soft silicone catheter with high biocompatibility ■ Colour-coded hub and label for easy size recognition ■ Urine drainage port fits KRUUSE urine bag ■ Radiopaque for x-ray visibility 																																																																																	
Sizes	<p>BUSTER HC Urinary Catheter</p> <table border="0" style="width: 100%;"> <tr> <td style="width: 15%;">Cat. No</td> <td style="width: 35%;">Size</td> <td style="width: 15%;">Colour</td> </tr> <tr> <td>273866</td> <td>1.6 mm x 50 cm / 5 Fr x 20 in</td> <td>●</td> </tr> <tr> <td>273867</td> <td>2.0 mm x 50 cm / 6 Fr x 20 in</td> <td>●</td> </tr> <tr> <td>273868</td> <td>2.6 mm x 50 cm / 8 Fr x 20 in</td> <td>●</td> </tr> <tr> <td>273869</td> <td>3.3 mm x 50 cm / 10 Fr x 20 in</td> <td>●</td> </tr> </table>	Cat. No	Size	Colour	273866	1.6 mm x 50 cm / 5 Fr x 20 in	●	273867	2.0 mm x 50 cm / 6 Fr x 20 in	●	273868	2.6 mm x 50 cm / 8 Fr x 20 in	●	273869	3.3 mm x 50 cm / 10 Fr x 20 in	●	<p>BUSTER Male Dog Catheter</p> <table border="0" style="width: 100%;"> <tr> <td style="width: 15%;">Cat. No</td> <td style="width: 35%;">Size</td> <td style="width: 15%;">Colour</td> </tr> <tr> <td>273405</td> <td>1.3 mm x 50 cm / 4 Fr x 20 in</td> <td>●</td> </tr> <tr> <td>273406</td> <td>1.6 mm x 50 cm / 5 Fr x 20 in</td> <td>●</td> </tr> <tr> <td>273410</td> <td>2.0 mm x 50 cm / 6 Fr x 20 in</td> <td>●</td> </tr> <tr> <td>273411</td> <td>2.3 mm x 50 cm / 7 Fr x 20 in</td> <td>●</td> </tr> <tr> <td>273420</td> <td>2.6 mm x 50 cm / 8 Fr x 20 in</td> <td>●</td> </tr> <tr> <td>273421</td> <td>3.0 mm x 50 cm / 9 Fr x 20 in</td> <td>●</td> </tr> <tr> <td>273430</td> <td>3.3 mm x 50 cm / 10 Fr x 20 in</td> <td>●</td> </tr> </table>	Cat. No	Size	Colour	273405	1.3 mm x 50 cm / 4 Fr x 20 in	●	273406	1.6 mm x 50 cm / 5 Fr x 20 in	●	273410	2.0 mm x 50 cm / 6 Fr x 20 in	●	273411	2.3 mm x 50 cm / 7 Fr x 20 in	●	273420	2.6 mm x 50 cm / 8 Fr x 20 in	●	273421	3.0 mm x 50 cm / 9 Fr x 20 in	●	273430	3.3 mm x 50 cm / 10 Fr x 20 in	●	<p>BUSTER NC Urinary Catheter</p> <table border="0" style="width: 100%;"> <tr> <td style="width: 15%;">Cat. No</td> <td style="width: 35%;">Size</td> <td style="width: 15%;">Colour</td> </tr> <tr> <td>273875</td> <td>1.6 mm x 50 cm / 5 Fr x 20 in</td> <td>●</td> </tr> <tr> <td>273876</td> <td>2.0 mm x 50 cm / 6 Fr x 20 in</td> <td>●</td> </tr> <tr> <td>273877</td> <td>2.6 mm x 50 cm / 8 Fr x 20 in</td> <td>●</td> </tr> <tr> <td>273878</td> <td>3.3 mm x 50 cm / 10 Fr x 20 in</td> <td>●</td> </tr> </table>	Cat. No	Size	Colour	273875	1.6 mm x 50 cm / 5 Fr x 20 in	●	273876	2.0 mm x 50 cm / 6 Fr x 20 in	●	273877	2.6 mm x 50 cm / 8 Fr x 20 in	●	273878	3.3 mm x 50 cm / 10 Fr x 20 in	●	<p>BUSTER Foley Catheter</p> <table border="0" style="width: 100%;"> <tr> <td style="width: 15%;">Cat. No</td> <td style="width: 35%;">Size</td> <td style="width: 15%;">Colour</td> </tr> <tr> <td>273860</td> <td>2 mm x 30 cm / 6 Fr x 12 in</td> <td>●</td> </tr> <tr> <td>273861</td> <td>2 mm x 55 cm / 6 Fr x 22 in</td> <td>●</td> </tr> <tr> <td>273862</td> <td>2.6 mm x 30 cm / 8 Fr x 12 in</td> <td>●</td> </tr> <tr> <td>273863</td> <td>2.6 mm x 55 cm / 8 Fr x 22 in</td> <td>●</td> </tr> <tr> <td>273858</td> <td>2.6 mm x 70 cm / 8 Fr x 28 in</td> <td>●</td> </tr> <tr> <td>273864</td> <td>3.3 mm x 30 cm / 10 Fr x 12 in</td> <td>●</td> </tr> <tr> <td>273865</td> <td>3.3 mm x 55 cm / 10 Fr x 22 in</td> <td>●</td> </tr> <tr> <td>273859</td> <td>3.3 mm x 70 cm / 10 Fr x 28 in</td> <td>●</td> </tr> </table>	Cat. No	Size	Colour	273860	2 mm x 30 cm / 6 Fr x 12 in	●	273861	2 mm x 55 cm / 6 Fr x 22 in	●	273862	2.6 mm x 30 cm / 8 Fr x 12 in	●	273863	2.6 mm x 55 cm / 8 Fr x 22 in	●	273858	2.6 mm x 70 cm / 8 Fr x 28 in	●	273864	3.3 mm x 30 cm / 10 Fr x 12 in	●	273865	3.3 mm x 55 cm / 10 Fr x 22 in	●	273859	3.3 mm x 70 cm / 10 Fr x 28 in	●
Cat. No	Size	Colour																																																																																			
273866	1.6 mm x 50 cm / 5 Fr x 20 in	●																																																																																			
273867	2.0 mm x 50 cm / 6 Fr x 20 in	●																																																																																			
273868	2.6 mm x 50 cm / 8 Fr x 20 in	●																																																																																			
273869	3.3 mm x 50 cm / 10 Fr x 20 in	●																																																																																			
Cat. No	Size	Colour																																																																																			
273405	1.3 mm x 50 cm / 4 Fr x 20 in	●																																																																																			
273406	1.6 mm x 50 cm / 5 Fr x 20 in	●																																																																																			
273410	2.0 mm x 50 cm / 6 Fr x 20 in	●																																																																																			
273411	2.3 mm x 50 cm / 7 Fr x 20 in	●																																																																																			
273420	2.6 mm x 50 cm / 8 Fr x 20 in	●																																																																																			
273421	3.0 mm x 50 cm / 9 Fr x 20 in	●																																																																																			
273430	3.3 mm x 50 cm / 10 Fr x 20 in	●																																																																																			
Cat. No	Size	Colour																																																																																			
273875	1.6 mm x 50 cm / 5 Fr x 20 in	●																																																																																			
273876	2.0 mm x 50 cm / 6 Fr x 20 in	●																																																																																			
273877	2.6 mm x 50 cm / 8 Fr x 20 in	●																																																																																			
273878	3.3 mm x 50 cm / 10 Fr x 20 in	●																																																																																			
Cat. No	Size	Colour																																																																																			
273860	2 mm x 30 cm / 6 Fr x 12 in	●																																																																																			
273861	2 mm x 55 cm / 6 Fr x 22 in	●																																																																																			
273862	2.6 mm x 30 cm / 8 Fr x 12 in	●																																																																																			
273863	2.6 mm x 55 cm / 8 Fr x 22 in	●																																																																																			
273858	2.6 mm x 70 cm / 8 Fr x 28 in	●																																																																																			
273864	3.3 mm x 30 cm / 10 Fr x 12 in	●																																																																																			
273865	3.3 mm x 55 cm / 10 Fr x 22 in	●																																																																																			
273859	3.3 mm x 70 cm / 10 Fr x 28 in	●																																																																																			
Packaging	12/pk	12/pk	12/pk	5/pk																																																																																	
Related Products	<table border="0" style="width: 100%;"> <tr> <td style="width: 15%;">Cat. No</td> <td style="width: 85%;">Description</td> </tr> <tr> <td>290798</td> <td>KRUUSE Urine Strips, 100/pk</td> </tr> <tr> <td>290916</td> <td>Refractometer</td> </tr> </table>	Cat. No	Description	290798	KRUUSE Urine Strips, 100/pk	290916	Refractometer	<table border="0" style="width: 100%;"> <tr> <td style="width: 15%;">Cat. No</td> <td style="width: 85%;">Description</td> </tr> <tr> <td>290798</td> <td>KRUUSE Urine Strips, 100/pk</td> </tr> <tr> <td>290916</td> <td>Refractometer</td> </tr> <tr> <td>340625</td> <td>KRUUSE Sterile Gel, 250 ml</td> </tr> <tr> <td>273855</td> <td>BUSTER Urine Bags, Luer Lock, 10/pk</td> </tr> <tr> <td>273857</td> <td>BUSTER Hanger for Urine Bag, 10/pk</td> </tr> </table>	Cat. No	Description	290798	KRUUSE Urine Strips, 100/pk	290916	Refractometer	340625	KRUUSE Sterile Gel, 250 ml	273855	BUSTER Urine Bags, Luer Lock, 10/pk	273857	BUSTER Hanger for Urine Bag, 10/pk	<table border="0" style="width: 100%;"> <tr> <td style="width: 15%;">Cat. No</td> <td style="width: 85%;">Description</td> </tr> <tr> <td>290798</td> <td>KRUUSE Urine Strips, 100/pk</td> </tr> <tr> <td>290916</td> <td>Refractometer</td> </tr> <tr> <td>340625</td> <td>KRUUSE Sterile Gel, 250 ml</td> </tr> <tr> <td>273855</td> <td>BUSTER Urine Bags, Luer Lock, 10/pk</td> </tr> <tr> <td>273857</td> <td>BUSTER Hanger for Urine Bag, 10/pk</td> </tr> </table>	Cat. No	Description	290798	KRUUSE Urine Strips, 100/pk	290916	Refractometer	340625	KRUUSE Sterile Gel, 250 ml	273855	BUSTER Urine Bags, Luer Lock, 10/pk	273857	BUSTER Hanger for Urine Bag, 10/pk	<table border="0" style="width: 100%;"> <tr> <td style="width: 15%;">Cat. No</td> <td style="width: 85%;">Description</td> </tr> <tr> <td>290798</td> <td>KRUUSE Urine Strips, 100/pk</td> </tr> <tr> <td>290916</td> <td>Refractometer</td> </tr> <tr> <td>340625</td> <td>KRUUSE Sterile Gel, 250 ml</td> </tr> <tr> <td>273855</td> <td>BUSTER Urine Bags, Adapter for Foley, 10/pk</td> </tr> <tr> <td>273857</td> <td>BUSTER Hanger for Urine Bag, 10/pk</td> </tr> </table>	Cat. No	Description	290798	KRUUSE Urine Strips, 100/pk	290916	Refractometer	340625	KRUUSE Sterile Gel, 250 ml	273855	BUSTER Urine Bags, Adapter for Foley, 10/pk	273857	BUSTER Hanger for Urine Bag, 10/pk																																							
Cat. No	Description																																																																																				
290798	KRUUSE Urine Strips, 100/pk																																																																																				
290916	Refractometer																																																																																				
Cat. No	Description																																																																																				
290798	KRUUSE Urine Strips, 100/pk																																																																																				
290916	Refractometer																																																																																				
340625	KRUUSE Sterile Gel, 250 ml																																																																																				
273855	BUSTER Urine Bags, Luer Lock, 10/pk																																																																																				
273857	BUSTER Hanger for Urine Bag, 10/pk																																																																																				
Cat. No	Description																																																																																				
290798	KRUUSE Urine Strips, 100/pk																																																																																				
290916	Refractometer																																																																																				
340625	KRUUSE Sterile Gel, 250 ml																																																																																				
273855	BUSTER Urine Bags, Luer Lock, 10/pk																																																																																				
273857	BUSTER Hanger for Urine Bag, 10/pk																																																																																				
Cat. No	Description																																																																																				
290798	KRUUSE Urine Strips, 100/pk																																																																																				
290916	Refractometer																																																																																				
340625	KRUUSE Sterile Gel, 250 ml																																																																																				
273855	BUSTER Urine Bags, Adapter for Foley, 10/pk																																																																																				
273857	BUSTER Hanger for Urine Bag, 10/pk																																																																																				

BUSTER Cat Catheters, side holes

BUSTER Cat Catheters with closed end and side holes
 Ideal for urine sample collection, urine retention relief and bladder lavage.
 The catheter is made from a DEHP-free medical grade PE, with good biocompatible properties.

- The luer lock adapter and the handle of the stylet are moulded for premium quality
- All catheters have rounded ends to avoid tissue damage
- Handles on luer lock adapter for better grip
- Suture holes in the baseplate allows suturing to the prepuce
- 12/pk

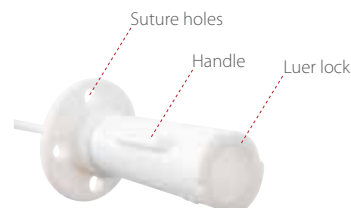

Cat. No	Description	Size, Metric	Size, Imperial	
273441	With stylet	1.0 x 100 mm	3 Fr x 4 in	
273451	Without stylet	1.0 x 130 mm	3 Fr x 5 1/8 in	
273461	Without stylet	1.3 x 130 mm	4 Fr x 5 1/8 in	
273453	With stylet	1.0 x 130 mm	3 Fr x 5 1/8 in	
273463	With stylet	1.3 x 130 mm	4 Fr x 5 1/8 in	
273457	Without stylet	1.3 x 160 mm	4 Fr x 6 1/8 in	NB extra long
273456	With stylet	1.3 x 160 mm	4 Fr x 6 1/8 in	NB extra long

BUSTER Cat Catheter with open end

The catheter with open end hole is useful for flushing uroliths retrograde into the bladder.
 The catheter is made from a DEHP-free medical grade PE, with good biocompatible properties.

- Extra rigid for easy introduction
- Handles on luer lock adapter for better grip
- Suture holes in the baseplate allows suturing to the prepuce
- 12/pk



Cat. No	Description	Size, Metric	Size, Imperial
273338	Without stylet	1.0 x 130 mm	3 Fr x 5 1/8 in
273339	Without stylet	1.3 x 130 mm	4 Fr x 5 1/8 in

BUSTER Easy Slide Cat Catheter



Popular cat catheter now available in BUSTER brand with the same good properties and then a little more...

Made from DEHP-free (phthalate free) medical grade of PTFE Polytetrafluoroethylene the BUSTER Easy Slide Cat catheter is available in two forms: with open end or side holes both suitable for drainage and/or bladder lavage.

Features and benefits

- High biocompatibility
- Suitable for medium-long placement (3 days)
- Optimal rigidity (no need for stylet), though still with proper softness
- Highly lubricious catheter tubing material that ensures easy catheterization
- Flexible baseplate with suturing holes allows securing of the catheter to the prepuce
- Luer Lock fitting which enables syringe-attachment for flushing and/or attaching a urine bag collection system to the catheter
- Radiopaque – visible in x-ray
- Individually packed in easy-open pouch
- Small package size

SMOOTH AND LUBRICIOUS FOR EASY AND UNCOMPLICATED CATHETERIZATION

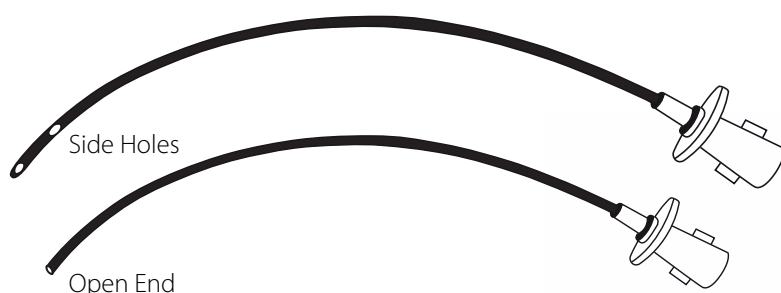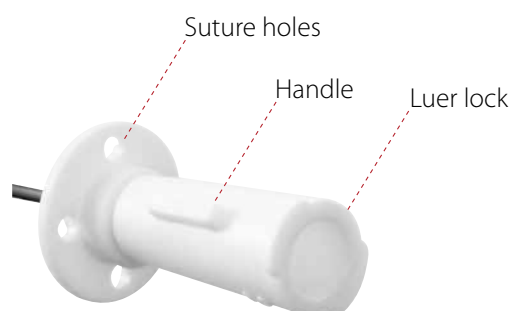

Cat. No	Type	Size, Metric	Size, Imperial	Package
273400	Side holes	1.2 x 110 mm	3.5 Fr x 4 1/2 in	5/pk
273401	Side holes	1.2 x 140 mm	3.5 Fr x 5 1/2 in	5/pk
273436	Open end	1.2 x 110 mm	3.5 Fr x 4 1/2 in	5/pk
273437	Open end	1.2 x 140 mm	3.5 Fr x 5 1/2 in	5/pk
273438	Open end	1.2 x 180 mm	3.5 Fr x 7 in	5/pk

BUSTER Barium Cat Catheter

The soft polyurethane catheter softens at body temperature and has the highest biocompatibility of all the cat catheters.

- Made of soft and biocompatible polyurethane and hence ideal for hospitalized patients
- Suitable for long term use in situ (up to 7 days)
- Suture holes in the baseplate allows suturing to the prepuce
- Female luer lock fitting: easy connection to syringe or urine collection bag
- X-Ray detectable which makes the catheter ideal for locating obstructions
- 12/pk



Cat. No	Description	Size, Metric	Size, Imperial
273467	Side holes, with stylet	1.0 x 130 mm	3 Fr x 5½ in
273468	Side holes, with stylet	1.3 x 130 mm	4 Fr x 5½ in





Period of Placement in Situ	Flushing and short term (0-1 day)	Medium term (1-3 days)	Long term (0-7 days)																																																																																				
Clinical Applications	<ul style="list-style-type: none"> Allows the vet or technician to flush uroliths retrograde into the bladder in the case of an obstruction Urine sample collection Check of passage in urethra Bladder tension relief Bladder lavage 	<ul style="list-style-type: none"> Secure urine passage. Typically used in FLUTD/FIC cats, hospitalized patients and patients undergoing surgery Urine sample collection Check of passage in urethra Bladder tension relief Bladder lavage 	<ul style="list-style-type: none"> Hospitalized patients and patients undergoing surgery Urine sample collection Check of passage in urethra Bladder tension relief Bladder lavage 																																																																																				
Features and Benefits	<ul style="list-style-type: none"> Holes in the base plate allow suturing to the prepuce Luer lock on hub for easy connection to syringe or urine bag 	<ul style="list-style-type: none"> Two lateral side holes and a rounded, atraumatically closed end Comes in various sizes with or without stylet Holes in the base plate allow suturing to the prepuce Luer lock on hub for easy connection to syringe or urine bag 	<ul style="list-style-type: none"> Two lateral side holes and a rounded, atraumatically closed end Holes in the base plate allow suturing to the prepuce Luer lock on hub for easy connection to syringe or urine bag 																																																																																				
Recommended Catheters	<p>BUSTER Easy Slide Cat Catheter, open end</p> <ul style="list-style-type: none"> Optimal rigidity for easy catheterization Softens and adjust to the anatomy Radiopaque for x-ray visibility <p>REMEMBER TO MONITOR URINE FLOW REGULARLY</p> <p>BUSTER Cat Catheter, open end</p> <ul style="list-style-type: none"> Soft and flexible catheter 	<p>BUSTER Easy Slide Cat Catheter, side holes</p> <ul style="list-style-type: none"> Optimal rigidity for easy catheterization Softens and adjust to the anatomy Radiopaque for x-ray visibility <p>BUSTER Cat Catheter, side holes</p> <ul style="list-style-type: none"> Soft and flexible catheter 	<p>BUSTER Barium Catheter</p> <ul style="list-style-type: none"> Made of polyurethane and contains barium Radiopaque for x-ray visibility Polyurethane softens at body temperature increasing the biocompatibility. Hence, it is suitable for longer term placement and hospitalized patients 																																																																																				
Sizes and Packaging	<p>BUSTER Easy Slide Cat Catheter, open end</p> <table border="0"> <tr> <td>Cat. No</td> <td>Description</td> <td>Size</td> <td>Package</td> </tr> <tr> <td>273436</td> <td>Without stylet</td> <td>1.2 x 110 mm / 3.5 Fr x 4¼ in</td> <td>5/pk</td> </tr> <tr> <td>273437</td> <td>Without stylet</td> <td>1.2 x 140 mm / 3.5 Fr x 5½ in</td> <td>5/pk</td> </tr> <tr> <td>273438</td> <td>Without stylet</td> <td>1.2 x 180 mm / 3.5 Fr x 7 in</td> <td>5/pk</td> </tr> </table> <p>BUSTER Cat Catheter, open end</p> <table border="0"> <tr> <td>Cat. No</td> <td>Description</td> <td>Size</td> <td>Package</td> </tr> <tr> <td>273338</td> <td>Without stylet</td> <td>1.0 x 130 mm / 3 Fr x 5¼ in</td> <td>12/pk</td> </tr> <tr> <td>273339</td> <td>Without stylet</td> <td>1.3 x 130 mm / 4 Fr x 5¼ in</td> <td>12/pk</td> </tr> </table>	Cat. No	Description	Size	Package	273436	Without stylet	1.2 x 110 mm / 3.5 Fr x 4¼ in	5/pk	273437	Without stylet	1.2 x 140 mm / 3.5 Fr x 5½ in	5/pk	273438	Without stylet	1.2 x 180 mm / 3.5 Fr x 7 in	5/pk	Cat. No	Description	Size	Package	273338	Without stylet	1.0 x 130 mm / 3 Fr x 5¼ in	12/pk	273339	Without stylet	1.3 x 130 mm / 4 Fr x 5¼ in	12/pk	<p>BUSTER Easy Slide Cat Catheter, side holes</p> <table border="0"> <tr> <td>Cat. No</td> <td>Description</td> <td>Size</td> <td>Package</td> </tr> <tr> <td>273400</td> <td>Without stylet</td> <td>1.2 x 110 mm / 3.5 Fr x 4¼ in</td> <td>5/pk</td> </tr> <tr> <td>273401</td> <td>Without stylet</td> <td>1.2 x 140 mm / 3.5 Fr x 5½ in</td> <td>5/pk</td> </tr> </table> <p>BUSTER Cat Catheter, side holes</p> <table border="0"> <tr> <td>Cat. No</td> <td>Description</td> <td>Size</td> <td>Package</td> </tr> <tr> <td>273451</td> <td>Without stylet</td> <td>1.0 x 130 mm / 3 Fr x 5¼ in</td> <td>12/pk</td> </tr> <tr> <td>273461</td> <td>Without stylet</td> <td>1.3 x 130 mm / 4 Fr x 5¼ in</td> <td>12/pk</td> </tr> <tr> <td>273457</td> <td>Without stylet</td> <td>1.3 x 160 mm / 4 Fr x 6¼ in</td> <td>12/pk</td> </tr> <tr> <td>273441</td> <td>With stylet</td> <td>1.0 x 100 mm / 3 Fr x 4 in</td> <td>12/pk</td> </tr> <tr> <td>273453</td> <td>With stylet</td> <td>1.0 x 130 mm / 3 Fr x 5¼ in</td> <td>12/pk</td> </tr> <tr> <td>273463</td> <td>With stylet</td> <td>1.3 x 130 mm / 4 Fr x 5¼ in</td> <td>12/pk</td> </tr> <tr> <td>273456</td> <td>With stylet</td> <td>1.3 x 160 mm / 4 Fr x 6¼ in</td> <td>12/pk</td> </tr> </table>	Cat. No	Description	Size	Package	273400	Without stylet	1.2 x 110 mm / 3.5 Fr x 4¼ in	5/pk	273401	Without stylet	1.2 x 140 mm / 3.5 Fr x 5½ in	5/pk	Cat. No	Description	Size	Package	273451	Without stylet	1.0 x 130 mm / 3 Fr x 5¼ in	12/pk	273461	Without stylet	1.3 x 130 mm / 4 Fr x 5¼ in	12/pk	273457	Without stylet	1.3 x 160 mm / 4 Fr x 6¼ in	12/pk	273441	With stylet	1.0 x 100 mm / 3 Fr x 4 in	12/pk	273453	With stylet	1.0 x 130 mm / 3 Fr x 5¼ in	12/pk	273463	With stylet	1.3 x 130 mm / 4 Fr x 5¼ in	12/pk	273456	With stylet	1.3 x 160 mm / 4 Fr x 6¼ in	12/pk	<p>BUSTER Barium Cat Catheter</p> <table border="0"> <tr> <td>Cat. No</td> <td>Description</td> <td>Size</td> <td>Package</td> </tr> <tr> <td>273467</td> <td>With stylet</td> <td>1.0 x 130 mm / 3 Fr x 5¼ in</td> <td>12/pk</td> </tr> <tr> <td>273468</td> <td>With stylet</td> <td>1.3 x 130 mm / 4 Fr x 5¼ in</td> <td>12/pk</td> </tr> </table>	Cat. No	Description	Size	Package	273467	With stylet	1.0 x 130 mm / 3 Fr x 5¼ in	12/pk	273468	With stylet	1.3 x 130 mm / 4 Fr x 5¼ in	12/pk
Cat. No	Description	Size	Package																																																																																				
273436	Without stylet	1.2 x 110 mm / 3.5 Fr x 4¼ in	5/pk																																																																																				
273437	Without stylet	1.2 x 140 mm / 3.5 Fr x 5½ in	5/pk																																																																																				
273438	Without stylet	1.2 x 180 mm / 3.5 Fr x 7 in	5/pk																																																																																				
Cat. No	Description	Size	Package																																																																																				
273338	Without stylet	1.0 x 130 mm / 3 Fr x 5¼ in	12/pk																																																																																				
273339	Without stylet	1.3 x 130 mm / 4 Fr x 5¼ in	12/pk																																																																																				
Cat. No	Description	Size	Package																																																																																				
273400	Without stylet	1.2 x 110 mm / 3.5 Fr x 4¼ in	5/pk																																																																																				
273401	Without stylet	1.2 x 140 mm / 3.5 Fr x 5½ in	5/pk																																																																																				
Cat. No	Description	Size	Package																																																																																				
273451	Without stylet	1.0 x 130 mm / 3 Fr x 5¼ in	12/pk																																																																																				
273461	Without stylet	1.3 x 130 mm / 4 Fr x 5¼ in	12/pk																																																																																				
273457	Without stylet	1.3 x 160 mm / 4 Fr x 6¼ in	12/pk																																																																																				
273441	With stylet	1.0 x 100 mm / 3 Fr x 4 in	12/pk																																																																																				
273453	With stylet	1.0 x 130 mm / 3 Fr x 5¼ in	12/pk																																																																																				
273463	With stylet	1.3 x 130 mm / 4 Fr x 5¼ in	12/pk																																																																																				
273456	With stylet	1.3 x 160 mm / 4 Fr x 6¼ in	12/pk																																																																																				
Cat. No	Description	Size	Package																																																																																				
273467	With stylet	1.0 x 130 mm / 3 Fr x 5¼ in	12/pk																																																																																				
273468	With stylet	1.3 x 130 mm / 4 Fr x 5¼ in	12/pk																																																																																				
Related Products	<table border="0"> <tr> <td>Cat. No</td> <td>Description</td> </tr> <tr> <td>290798</td> <td>KRUUSE Urine Strips, 100/pk</td> </tr> <tr> <td>290916</td> <td>Refractometer</td> </tr> <tr> <td>340625</td> <td>KRUUSE Sterile Gel, 250 ml</td> </tr> </table>	Cat. No	Description	290798	KRUUSE Urine Strips, 100/pk	290916	Refractometer	340625	KRUUSE Sterile Gel, 250 ml	<table border="0"> <tr> <td>Cat. No</td> <td>Description</td> </tr> <tr> <td>290798</td> <td>KRUUSE Urine Strips, 100/pk</td> </tr> <tr> <td>290916</td> <td>Refractometer</td> </tr> <tr> <td>340625</td> <td>KRUUSE Sterile Gel, 250 ml</td> </tr> <tr> <td>273855</td> <td>BUSTER Urine Bags, Luer Lock, 10/pk</td> </tr> <tr> <td>273857</td> <td>BUSTER Hanger for Urine Bag, 10/pk</td> </tr> </table>	Cat. No	Description	290798	KRUUSE Urine Strips, 100/pk	290916	Refractometer	340625	KRUUSE Sterile Gel, 250 ml	273855	BUSTER Urine Bags, Luer Lock, 10/pk	273857	BUSTER Hanger for Urine Bag, 10/pk	<table border="0"> <tr> <td>Cat. No</td> <td>Description</td> </tr> <tr> <td>290798</td> <td>KRUUSE Urine Strips, 100/pk</td> </tr> <tr> <td>290916</td> <td>Refractometer</td> </tr> <tr> <td>340625</td> <td>KRUUSE Sterile Gel, 250 ml</td> </tr> <tr> <td>273855</td> <td>BUSTER Urine Bags, Luer Lock, 10/pk</td> </tr> <tr> <td>273857</td> <td>BUSTER Hanger for Urine Bag, 10/pk</td> </tr> </table>	Cat. No	Description	290798	KRUUSE Urine Strips, 100/pk	290916	Refractometer	340625	KRUUSE Sterile Gel, 250 ml	273855	BUSTER Urine Bags, Luer Lock, 10/pk	273857	BUSTER Hanger for Urine Bag, 10/pk																																																				
Cat. No	Description																																																																																						
290798	KRUUSE Urine Strips, 100/pk																																																																																						
290916	Refractometer																																																																																						
340625	KRUUSE Sterile Gel, 250 ml																																																																																						
Cat. No	Description																																																																																						
290798	KRUUSE Urine Strips, 100/pk																																																																																						
290916	Refractometer																																																																																						
340625	KRUUSE Sterile Gel, 250 ml																																																																																						
273855	BUSTER Urine Bags, Luer Lock, 10/pk																																																																																						
273857	BUSTER Hanger for Urine Bag, 10/pk																																																																																						
Cat. No	Description																																																																																						
290798	KRUUSE Urine Strips, 100/pk																																																																																						
290916	Refractometer																																																																																						
340625	KRUUSE Sterile Gel, 250 ml																																																																																						
273855	BUSTER Urine Bags, Luer Lock, 10/pk																																																																																						
273857	BUSTER Hanger for Urine Bag, 10/pk																																																																																						

Sum up



BUSTER HC Urinary Catheter -Hydrophilic Coated

- Hydrophilic coated PVC catheter
- Ideal for short term use
- Reduces the friction and discomfort during catheterization

BUSTER NC Urinary Catheter -Non Coated

- Good biocompatible PVC catheter
- Flexible and soft for comfortable catheterization
- User-friendly end-opening box

BUSTER Dog Catheter

- Rigid PP tubing for easy introduction
- Biggest assortment world-wide
- User-friendly blister packaging

BUSTER Foley Catheter, Silicone

- Silicone catheter with very high biocompatibility
- Suitable for long term placement
- Retention balloon to secure appropriate placement of the catheter



BUSTER Cat Catheter with closed end and side holes

- Biocompatible PE catheter
- Popular catheter with closed end and side holes
- Suture holes in baseplate
- Luer Lock and handles for better grip

BUSTER Cat Catheter with open end

- Biocompatible PE catheter
- More Rigid
- Suitable for flushing uroliths retrograde

BUSTER Easy Slide Cat Catheter

- Biocompatible PTFE catheter
- Optimal rigidity (no need for stylet), though still with proper softness
- Flexible baseplate with suturing holes allows securing of the catheter to the prepuce
- Radiopaque – visible in x-ray

BUSTER Barium Cat Catheter

- Soft, biocompatible PU catheter
- Softens at body temperature
- X-ray detectable




Related Products

BUSTER Urine Bags, 750 ml, Luer Lock Connection

Connect the urine bag to a Buster Catheter or BUSTER NC Urinary catheter to collect and control urine flow during catheterization. The bag has a non-return valve, which prevent the urine from running retrograde. Further the bag has an indicative volume scale and a "push-pull" release valve, which makes it possible to empty the bag.

Cat. No 273855
10/pk

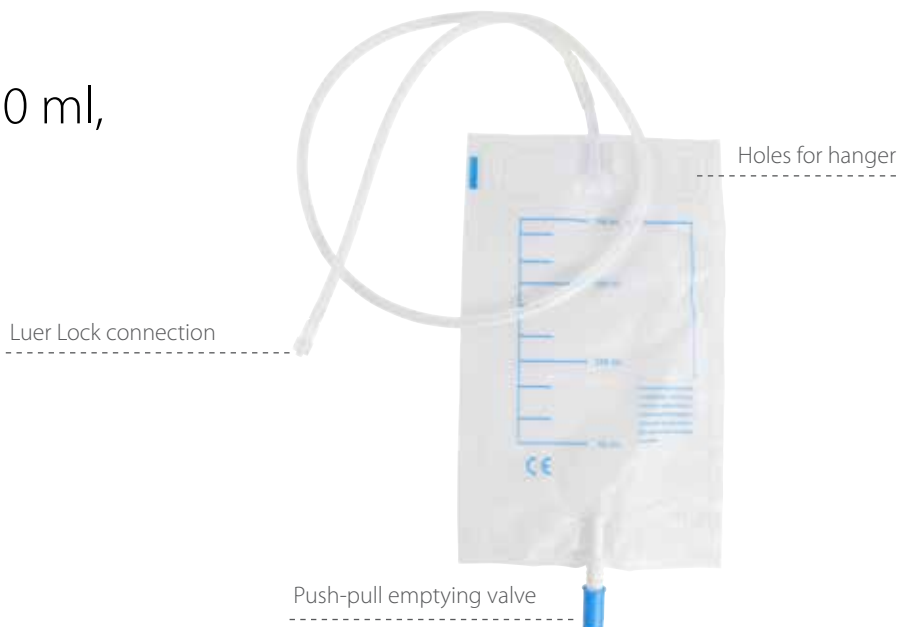

BUSTER Urine Bags, 750 ml, Adapter for Foley Catheter

Connect the urine bag to BUSTER Foley Catheter to collect and control urine flow during catheterization. The bag has a non-return valve, which prevent the urine from running retrograde. Further the bag has an indicative volume scale and a "push-pull" release valve, which makes it possible to empty the bag.

Cat. No 273856
10/pk



BUSTER Hanger for Urine Bags

Hanger for BUSTER Urine bags (Cat. No 273855 and 273856).
For easy handling of BUSTER Urine bags.
Easy to hang on surgery table or on the cage.

Cat. No 273857
10/pk



KRUUSE Urine Strips

Inexpensive and easy-to-read test. Coloured indicator squares on the strip can be compared with the colour chart on the label. Results appear in 60 seconds. Parameters: pH, nitrites, protein, glucose, ketones, bilirubin, urobilinogen, blood, leukocytes, specific gravity (For specific gravity we advise to use our refractometers).

Cat. No 290798
100/pk



Refractometer Specific gravity

For measuring the specific gravity in urine.

Normal range:
Dog: 1.015 - 1.040
Cat: 1.015 - 1.050

Cat. No 290916



Uri Pet

Uripet Urine sample collection
Easy devise for collecting urine sample from bitches.

Cat. No 273710
1/pk



Catrine Litter Pearls

Catrine Urine collection kit

This is a convenient and inexpensive kit for collecting urine samples from cats, developed specifically for the veterinarian and the cat owner.

Cat. No 275021



Kit4Cat

Kit4Cat, sealsand 3 x 300 gr

Kit4Cat is revolutionary biodegradable hydrophobic sand that has a natural non-toxic urine repelling coating.

It allows veterinarians, technicians, and cat owners to quickly and easily collect a cat's urine sample, without resorting to invasive procedures such as Cystocentesis.

The cat's urine will stay on top, allowing you to collect the urine using the pipette and vial that are included in every bag.

Cat. No 275011



Gel

BOVIVET gel 500 ml

Lubricating gel.

Apply gel on the catheter in order to make catheterization easier and more comfortable.

Cat. No 180520

KRUUSE Steril gel 250 ml

Lubricating gel.

Apply gel on the catheter in order to make catheterization easier and more comfortable.

Cat. No 340625







JØRGEN KRUUSE A/S
Havretoften 4
DK-5550 Langeskov - Denmark
Tel.: +45 72 14 14 14
Fax: +45 72 14 15 00
www.kruuse.com