

KRUUSE Soft-Flex

Two unique kinds of very soft bandages for fixation of dressings and controlled support of sprains and distortions.

Strong, comfortable and user-friendly. Easy to tear.

- Same bandage - two different adhesives
- Very soft and comfortable for the patient
- Non-slip, gives good fixation of bandages
- No shrinkage



Features and benefits

Very comfortable

KRUUSE Soft-Flex is a white 5 mm soft and light weight polyurethane foam bandage.

Choice of adhesiveness

KRUUSE Soft-Flex Adhesive with a strong adhesive backing
KRUUSE Soft-Flex Cohesive, sticks to it-self, not to skin and hair.

Easy to apply

KRUUSE Soft-Flex bandages are supplied with back releasing paper through the complete roll.

Skin friendly

KRUUSE Soft-Flex is porous and permits air-flow, which gives a comfortable environment on the skin.
Leaves no residue upon removal.

Reliable and stable fixation

The foam bandage is a 5 mm non-slip material, which makes the bandage strong and firm yet comfortable.

Skin friendly

KRUUSE Vet-Flex Cohesive is coated with latex, which makes the cohesiveness that sticks to it-self, but not to other surfaces like skin or hair.

Good compliance from the patient

The bandage is very light, does not constrict and is not sticky on the outside of the bandage.

Stable foam

The foam stays stable and does not collapse when used as polster e.g.



Back releasing paper



Cohesive on both sides



Adhesive at one side

Recommended use for KRUUSE Soft-Flex bandages

- Controlled support of joints, sprains, distortions
- Fixating dressings
- As polster underneath compression bandages or cast

Instructions for use

Application

Evaluate to use KRUUSE Soft-Flex Cohesive or Adhesive taking skin, hair and wound into consideration.

Fixate the end of the bandage and apply evenly over a larger area of the injured body part according to the level of compression the area needs.

When application is done, cut or tear the bandage.

When applied, squeeze gently the layers together to enhance the cohesiveness/adhesiveness.

Use KRUUSE Soft-Flex as secondary/tertiary bandage on top of primary/secondary dressing.

Primary dressing suggestions:

- KRUUSE Hydro Gel
- KRUUSE Manuka Honey

Secondary dressing suggestion:

- KRUUSE Foam Dressing

Can also be used in combination with

- KRUUSE Vet-Plast
- KRUUSE Vet-Flex
- BUSTER Tubular Bandage

Removal

Cut gently with a bandage scissor, be careful not to cut into the skin.

Do not use

If patient or user is allergic to latex (KRUUSE Soft-Flex Cohesive only).

Be aware of compression during application, if the bandage is applied with too much tension or with too many layers over a small part of the body. Apply evenly over a larger area.

Cat. No	Description
160963	KRUUSE Soft-flex Adhesive 5 cm x 2.5 m
160964	KRUUSE Soft-flex Adhesive 10 cm x 2.5 m
160965	KRUUSE Soft-flex Adhesive 15 cm x 2.5 m
160966	KRUUSE Soft-flex Cohesive 5 cm x 2.5 m
160967	KRUUSE Soft-flex Cohesive 10 cm x 2.5 m
160968	KRUUSE Soft-flex Cohesive 15 cm x 2.5 m