



KRUUSE Sacryl Rapid

Multifilament, braided: For secure knot tying, high flexibility and easy handling

Coated: For easy knot tying and smooth passage through tissue

Irradiation: By gamma irradiation to accommodate rapid absorption

Absorbable: By hydrolysis

KRUUSE Sacryl Rapid is a braided and coated multifilament synthetic suture composed of Polyglactin 910.

KRUUSE Sacryl Rapid meets all the requirements established by the European Pharmacopoeia (EP) and the United States Pharmacopoeia (USP) for Synthetic Absorbable Surgical Sutures.

Due to the rapid loss of its breaking strength KRUUSE Sacryl Rapid should not be used where extended approximation of tissue under stress is required, such as cardiovascular tissue or muscle, or where wound support beyond 7 days is required.

The characteristics of rapid loss of strength are achieved by use of gamma irradiation.

Absorption: Absorbable:

- Complete mass absorption in approximately 45 days
- Absorption by hydrolysis

Chemical composition:

- KRUUSE Sacryl Rapid is composed of Polyglactin 910 (copolymer of 90% Glycolide and 10% L-lactide)

Tensile Strength:

- 55 % remains after 5 days
- 0 % remains after 10-14 days
- Complete mass absorption in approximately 45 days

Colour:

- Undyed

Indications:

KRUUSE Sacryl Rapid is suitable for use in soft tissue approximation where short term wound support and rapid absorption of the suture is indicated, including:

- Superficial skin closure, both subcuticular and cuticular
- Suturing of conjunctiva in ophthalmic surgery
- Suturing of oral cavity mucosa
- Episiotomies

Contraindications:

- Where longer or permanent support is required

Range:

- USP 5-0 to 3-0
- EP 1 to 2

Sterilization method:

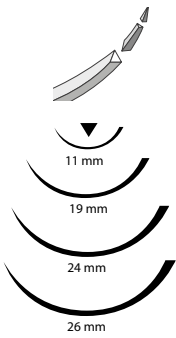

- EO

Packing:

- Boxes of 18 peel packs



KRUUSE Sacryl Rapid

Shape of needle	Needle length mm	USP	EP	Length in cm	Suture colour	Number per box	Cat. No
3/8 circle, reverse cutting 	11	5-0	1	45	Undyed	18	152356
	19	5-0	1	45	Undyed	18	152352
	19	4-0	1.5	45	Undyed	18	152351
	19	3-0	2	70	Undyed	18	152354
	24	3-0	2	70	Undyed	18	152350
	26	2-0	3	70	Undyed	18	152357
3/8 circle, conventional cutting 	16	4-0	1.5	70	Undyed	18	152353
	16	3-0	2	70	Undyed	18	152355

For more information visit:
www.kruuse.com

