



Mainatvet
by **KRUUSE**

Equine Garments




KRUUSE Recovery Bandaging

3-dimensional elastic wrap which ends in a wide foam part covered in velour adhesive to TRIHOOK* (*Extra strong adhesive strip). Interior pocket lined with a monofilm pad 1 cm thick which acts as an air cushion covering the lower abdomen.


- Makes it easier for the horse to stand up
- Full protection in case of violent recovery
- Maximum immediate post-operative protection for the first hours after the surgery

Cat. No	Description	Size	Circumference
165200	KRUUSE Foal Recovery Bandaging	S	70 - 89 cm
165201	KRUUSE Foal Recovery Bandaging	M	90 - 109 cm
165202	KRUUSE Foal Recovery Bandaging	L	110 - 130 cm
165203	KRUUSE Equine Recovery Bandaging	S	130- 170 cm
165204	KRUUSE Equine Recovery Bandaging,	M	170 - 200 cm
165205	KRUUSE Equine Recovery Bandaging	L	200 - 240 cm

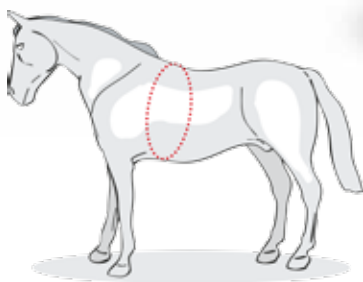

How to measure the size

How to apply KRUUSE Recovery Bandaging

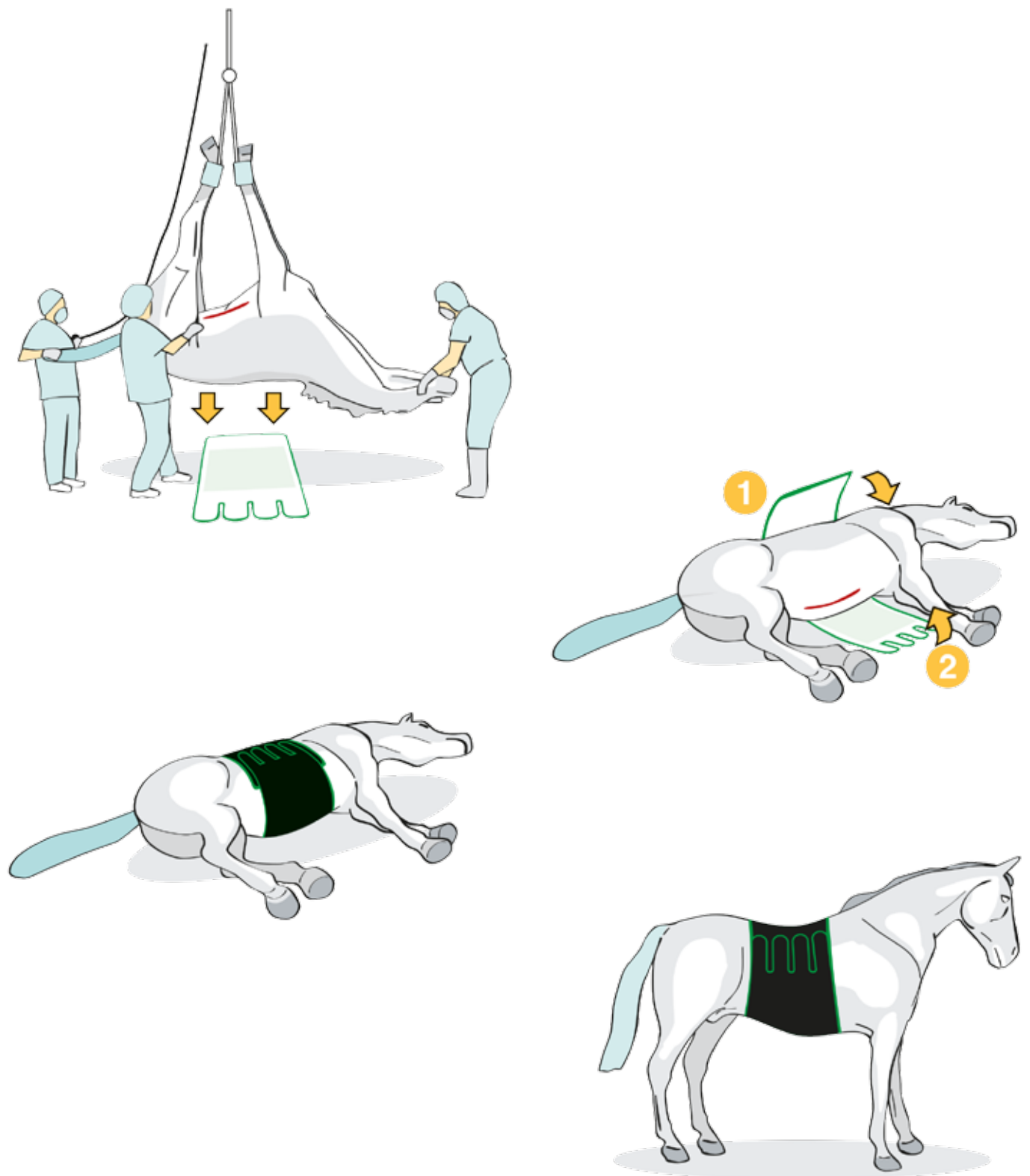

One of the first hurdles the horse encounters post laparotomy is the anaesthetic recovery where it is paramount the incision is protected from external trauma and potential contamination. The Kruuse Recovery Bandages are used on every colic patient in our hospital and achieve both these goals. We find them easy to apply and are secure throughout anaesthetic recovery. The Post Colic Surgery bandages are useful for laparotomy incisions where extra attention is required as they provide additional support are quick and easy to change.

Neil Townsend MSc BVSc Cert ES (Soft Tissue) DipECVS MRCVS
RCVS and European Specialist in Equine Surgery
University of Liverpool

KRUUSE Post Colic Surgery Kit

Fit in less than three minutes - the perfect postoperative bandaging


Outside: foam covered in velour adhesive to Trihook *(*extra strong adhesive strip)

Inside: 100% skin friendly cotton. Breathable. Finishing with elastic ribbon. No sores or friction points

The kit includes

- 1 abdominal wrap with a removable pocket which is washable and the mesh of which can be slit to replace absorbent drainage pad inside
- 2 replacement interior pockets
- 3-4 dorsal straps (depending on the size)
- 1 breast strap

Cat. No	Description	Size	Circumference
165206	KRUUSE Foal Post Colic Surgery Kit	S	70 - 89 cm
165207	KRUUSE Foal Post Colic Surgery Kit	M	90 - 109 cm
165208	KRUUSE Foal Post Colic Surgery Kit	L	110 - 130 cm
165209	KRUUSE Equine Post Colic Surgery Kit	S	130- 170 cm
165210	KRUUSE Equine Post Colic Surgery Kit	M	170 - 200 cm
165211	KRUUSE Equine Post Colic Surgery Kit	L	200 - 240 cm

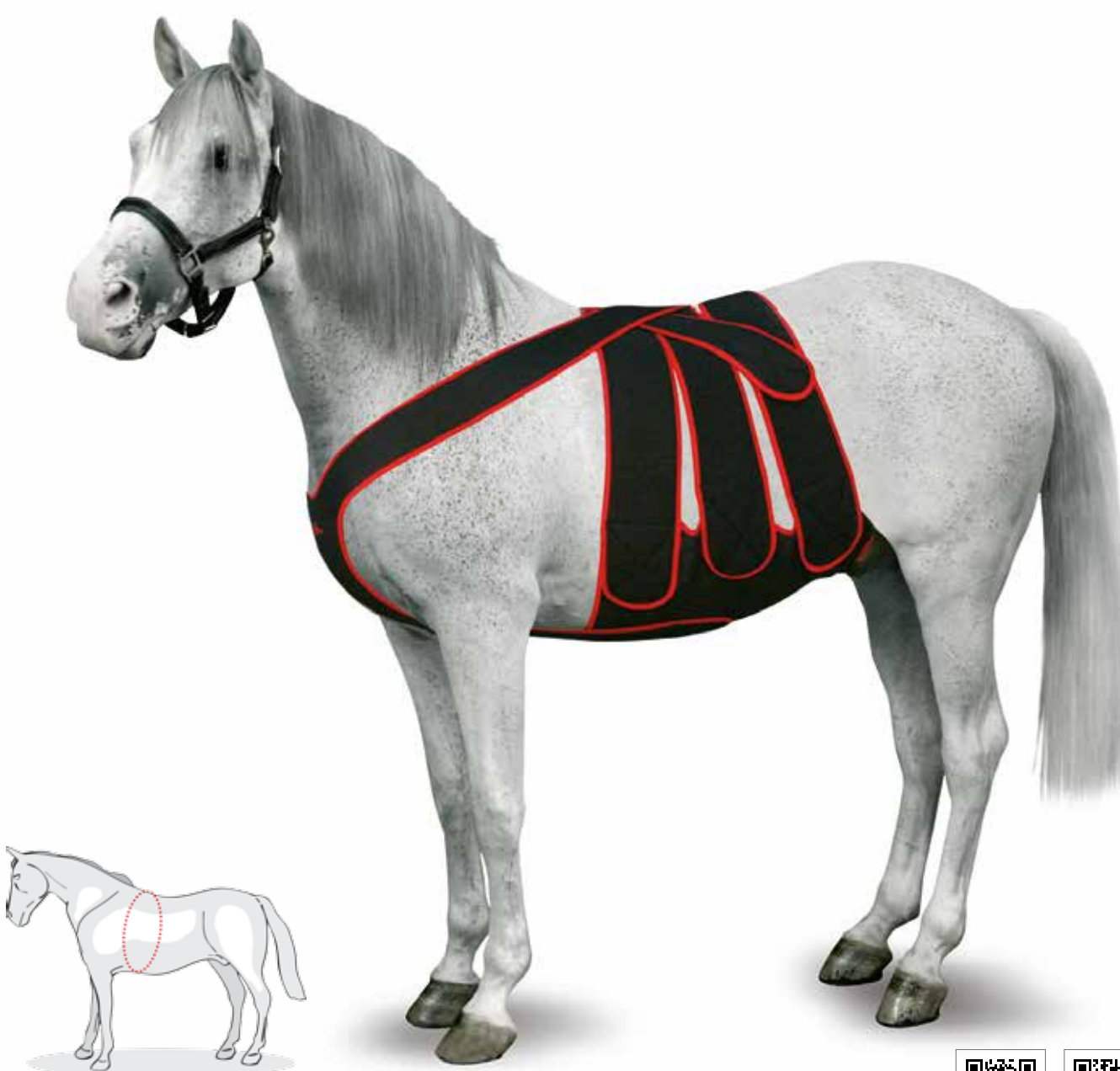

How to measure the size

Scan QR code
to see video



Application notes



Indication

Abdominal surgery.

Technical specifications

Abdominal wrap with a removable interior pocket which is washable and the mesh of which can be slit to replace absorbant drainage pad inside. Replacements come in packets of 12 units.

The kit includes

1 abdominal wrap, 2-4 dorsal straps (depending on the size of horse), 1 breast strap and 2 replacement interior pockets.



Replacements

Cat. No	Description	Size
165323	KRUUSE Equine Breast Strap	S
165324	KRUUSE Equine Breast Strap	M
165325	KRUUSE Equine Breast Strap	L
165303	KRUUSE Foal Pockets 12/pk	S
165304	KRUUSE Foal Pockets 12/pk	M
165305	KRUUSE Foal Pockets 12/pk	L
165326	KRUUSE Equine Pockets 12/pk	S
165327	KRUUSE Equine Pockets 12/pk	M
165328	KRUUSE Equine Pockets 12/pk	L

KRUUSE Equine Scrotum Strap

This strap is an addition to the Post Colic Surgery Kit and only for male horses.

The strap has three adjustment points with the goal to adjust the little hollow in front of the penis to avoid that the horse urinate inside the abdominal wrap.

Use : Fit the adjustment in the middle just in front of the penis and fasten on the dorsal strap.

Cat. No	Description	Size
165300	KRUUSE Foal Scrotum Strap	S
165301	KRUUSE Foal Scrotum Strap	M
165302	KRUUSE Foal Scrotum Strap	L
165320	KRUUSE Equine Scrotum Strap	S
165321	KRUUSE Equine Scrotum Strap	M
165322	KRUUSE Equine Scrotum Strap	L



KRUUSE Frontal Bandaging

Combination of 3-dimensional elastic tissue with large adjustable areas of foam, which makes this bandaging indispensable in covering wounds and cuts in places difficult to bandage. Standard length of foreleg sleeves 22 cm.

Cat. No	Description	Size	Circumference
165212	KRUUSE Foal Frontal Bandaging	S	70 - 89 cm
165213	KRUUSE Foal Frontal Bandaging	M	90 - 109 cm
165214	KRUUSE Equine Frontal Bandaging	S	130- 170 cm
165215	KRUUSE Equine Frontal Bandaging	M	170 - 200 cm
165216	KRUUSE Equine Frontal Bandaging	L	200 - 240 cm



How to measure the the size

Scan QR code
to see video



KRUUSE Eye Masks

Please note that all eye masks are delivered with only one eye protector

KRUUSE Eye Mask with 1 Patch

The mask is held in place with adhesive elastic straps, adjustable to height and contours to avoid window edemas. Easy to use. The flat patch can be adjusted independently right or left on the mask.

Cat. No	Description	Size
165217	KRUUSE Foal Eye Mask w. 1 patch	S
165218	KRUUSE Foal Eye Mask w. 1 patch	M
165219	KRUUSE Equine Eye Mask w. 1 patch,	S
165220	KRUUSE Equine Eye Mask w. 1 patch	M
165221	KRUUSE Equine Eye Mask w. 1 patch	L

Spare parts

165312	KRUUSE Foal Eye Patch (165217)	S
165313	KRUUSE Foal Eye Patch (165218)	M
165338	KRUUSE Equine Eye Patch (165219)	S
165339	KRUUSE Equine Eye Patch (165220)	M
165340	KRUUSE Equine Eye Patch (165221)	L



KRUUSE Full Eye Protector Mask

The mask is held in place with adhesive elastic straps, adjustable to height and contours to avoid window edemas.

The semi-flexible full eye protector made of 4 mm low-density polyethylene can be adjusted without effort right or left on the mask without the need to remove the whole mask.

Cat. No	Description	Size
165222	KRUUSE Foal Full Eye Protector Mask	S
165223	KRUUSE Foal Full Eye Protector Mask	M
165224	KRUUSE Equine Full Eye Protector Mask	S
165225	KRUUSE Equine Full Eye Protector Mask	M
165226	KRUUSE Equine Full Eye Protector Mask	L

Spare parts

165306	KRUUSE Foal Spare Full Eye Protector (165222)	S
165307	KRUUSE Foal Spare Full Eye Protector (165223)	M
165329	KRUUSE Equine Spare Full Eye Protector (165224)	S
165330	KRUUSE Equine Spare Full Eye Protector (165225)	M
165331	KRUUSE Equine Spare Full Eye Protector (165226)	L



KRUUSE Full Perforated Eye Protector Mask

The mask is held in place with adhesive elastic straps, adjustable to height and contours to avoid window edemas.

The semi flexible full eye protector made of 4 mm low density polyethylene has eight perforations to optimize the air circulation.

The eye protector is adjusted on the mask left or right side without the need to remove the whole mask.

Cat. No	Description	Size
165227	KRUUSE Foal Full Perforated Eye Protector Mask	S
165228	KRUUSE Foal Full Perforated Eye Protector Mask	M
165229	KRUUSE Equine Full Perforated Eye Protector Mask	S
165230	KRUUSE Equine Full Perforated Eye Protector Mask	M
165231	KRUUSE Equine Full Perforated Eye Protector Mask	L

Spare parts

165308	KRUUSE Foal Spare Full Perforated Eye Protector (165227)	S
165309	KRUUSE Foal Spare Full Perforated Eye Protector (165228)	M
165332	KRUUSE Equine Spare Full Perforated Eye Protector (165229)	S
165333	KRUUSE Equine Spare Full Perforated Eye Protector (165230)	M
165334	KRUUSE Equine Spare Full Perforated Eye Protector (165231)	L



KRUUSE Half Eye Protector Mask

The mask is held in place with adhesive elastic straps, adjustable to height and contours to avoid window edemas.

The semi-flexible half eye protector made of 4 mm low density polyethylene can be adjusted independently right or left on the mask.

Cat. No	Description	Size
165232	KRUUSE Foal Half Eye Protector Mask	S
165233	KRUUSE Foal Half Eye Protector Mask	M
165234	KRUUSE Equine Half Eye Protector Mask	S
165235	KRUUSE Equine Half Eye Protector Mask	M
165236	KRUUSE Equine Half Eye Protector Mask	L

Spare parts

165310	KRUUSE Foal Spare Half Eye Protector (165232)	S
165311	KRUUSE Foal Spare Half Eye Protector (165233)	M
165335	KRUUSE Equine Spare Half Eye Protector (165234)	S
165336	KRUUSE Equine Spare Half Eye Protector (165235)	M
165337	KRUUSE Equine Spare Half Eye Protector (165236)	L




KRUUSE Televet Electrode Support

The KRUUSE Televet Electrode Support is developed to keep Televet and patient cables with electrodes in place and protected. It can be used in an ICU set-up as well as during lunging in an arena. The KRUUSE Televet Electrode Support is a dual band divided into three parts. One on each side to protect cables and electrodes and a middle flap for the Televet.

The KRUUSE Televet Electrode Support is fixed with a breast strap to ensure that it stays in place. A practical and safe way to use the Televet.

Cat. No	Description	Size
165237	KRUUSE Televet Electrode Support	M
165238	KRUUSE Televet Electrode Support	L

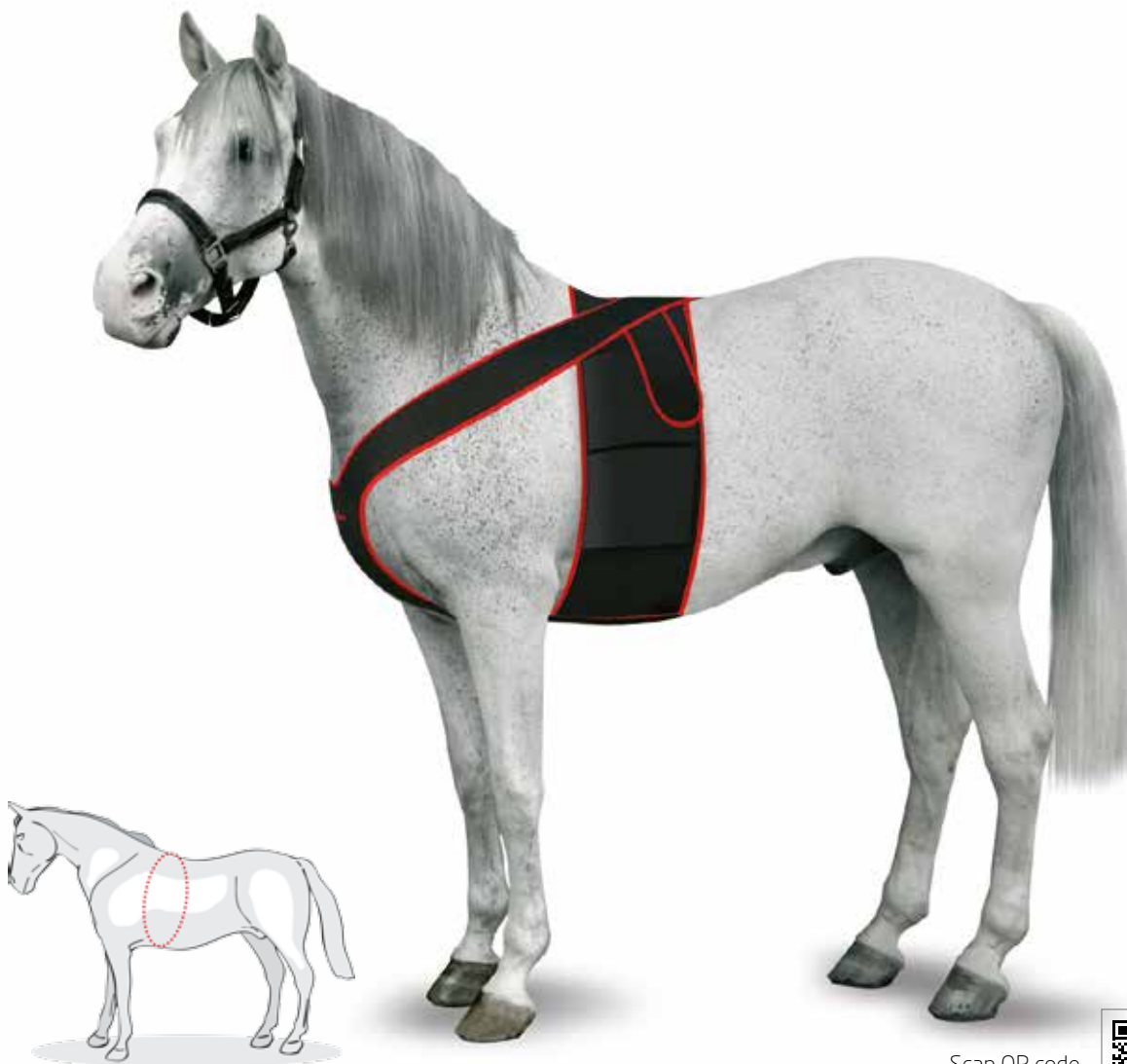

How to measure the size

Scan QR code
to see video



"As a user of the Televet system for many years the development of the recent KRUUSE Televet Electrodes Support has been a significant product addition for the Televet Kit. The new KRUUSE Televet Electrodes Support is excellent and I have been using this now for many months. It is both comfortable for the horse and ensures the electrodes stay in place particularly for 24hr testing. This is an excellent addition and I would recommend any Televet owner to invest in one."

Prof. Catherine McGowan

Head of Equine Internal Medicine and Director of Veterinary Postgraduate Education
University of Liverpool

JØRGEN KRUUSE A/S EXP.
Havretoften 4
DK-5550 Langeskov
Tlf.: (+45) 72 14 15 16
Fax: (+45) 72 14 15 00
info@kruuse.com

JØRGEN KRUUSE A/S DK
Havretoften 4
DK-5550 Langeskov
Tlf.: (+45) 72 14 14 14
Fax: (+45) 72 14 14 00
info@kruuse.com

KRUUSE SVENSKA AB
Ullsväg 30
756 51 Uppsala
Tlf.: (+46) 018-489 97 00
Fax: (+46) 018-489 97 98
kruuse.svenska@kruuse.com

KRUUSE NORGE AS
Dyrløkkeveien 19
N-1440 Drøbak
Tlf.: (+47) 64 90 75 00
Fax: (+47) 64 90 75 01
kruuse.norge@kruuse.com

KRUUSE UK LTD.
12 Sherburn Network Centre,
Lancaster Close, Sherburn in Elmet,
North Yorkshire, LS25 6NS.
Phone: (+44) 01977 681523
Fax: (+44) 01977 683537
kruuse.uk@kruuse.com

