KRUUSE MANUKA AD

Honey impregnated absorbent dressing
Primary wound dressing

100% Leptospermum
scoparium honey

GB: INSTRUCTIONS FOR USE

KRUUSE Manuka AD

Honey Impregnated Absorbent Dressing

A super absorbent polyacrylate polymer and polyester fiber dressing impregnated with 100% Leptospermum scoparium honey from New Zealand. Manuka AD has a hydrocolloid surface coating to improve the ease of handling and applying the dressing.

Indications for Use

- Traumatic and contaminated wounds
- Sloughy wounds
- Superficial and partial thickness burns
- Pressure sores
- Cavities and abscesses
- Surgical wounds

Directions for Use

DO NOT RE-USE. For single use.

Remove protective liners and apply directly to wound, either side down; cover with secondary dressing of choice.

Manuka AD may be cut to size. For large wounds use additional dressings placed side by side.

Cover with appropriate secondary dressing (e.g. a film or KRUUSE Foam dressing).

Side effects:

Temporary increased pain may be experienced due to osmotic action and/or the low pH of honey.

If pain persists, discontinue use of the dressing and gently irrigate the wound with sterile saline solution.

If re-used there is a high risk of infection and cross contamination.

Dressing change

Depending on wound exudate, any surrounding interstitial fluid, edema and dressing protocol, Manuka AD may require a daily application. In exuding wounds, the honey will be diluted by the exudate and absorbed into the secondary dressing. Re-apply if necessary.

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