# KRUUSE MANUKA G

Honey wound gel dressing

100% Leptospermum  
scoparium honey

## GB: INSTRUCTIONS FOR USE

### KRUUSE Manuka G, sterile honey

Manuka G consist of 100% Leptospermum Scoparium honey from New Zealand in a single dose applicator tube.

### Indications for Use

- Traumatic and contaminated wounds including minor abrasions and lacerations
- Sloughy wounds
- Superficial and partial thickness burns
- Cavity wounds including abscesses
- Pressure sores
- Surgical wounds

### Directions for Use

DO NOT RE-USE. For single use.

Snap-off cap Apply directly to wound or suitable primary dressing (e.g. an alginate, gauze, or absorbent pad). Cover with appropriate secondary dressing (e.g. a film or island dressing).

### Side effects

Temporary increased pain may be experienced due to osmotic action and/or the low pH of honey. If pain persists, discontinue use of the dressing and gently irrigate the wound with sterile saline solution. If re-used there is a high risk of infection and cross contamination.

### Dressing change

Depending on wound exudate, any surrounding interstitial fluid, edema and local dressing protocol, Manuka G may require a daily application. In exuding wounds, Manuka G will be diluted by the exudate and absorbed into the secondary dressing. Re-apply as necessary.

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