KRUUSE INSEMINATION GEL

For veterinary use only

250 ml
340624

KRUUSE INSEMINATION GEL

GB Product Description

This insemination gel is a medical-grade lubricant designed for use during insemination procedures and with urinary catheters, AI catheters, and medical instruments. The gel is **non-spermicidal**, contains **no preservatives**, and is composed of **Aqua** and lubricating agents (**Sodium Carboxymethylcellulose and Polyethylene Oxide**).

Intended Use

- Artificial insemination (AI)
- Urinary catheters
- AI catheters
- Medical instruments

Key Features

- Safe for sperm cells
- Excellent lubricating properties
- Free from preservatives and spermicides
- Compatible with medical devices

Directions for Use

1. **Hygiene:** Wash hands thoroughly before use.
2. **Preparation:** Ensure that the catheter or instrument is clean and ready for use.
3. **Application:**
 - Apply an appropriate amount of gel directly to the catheter or instrument
 - Alternatively, apply the gel to the area where the instrument will be inserted
4. **Procedure:** Follow standard procedures for insemination or catheterization.
5. **After Use:** Discard any excess gel and clean instruments according to manufacturer guidelines.

Important

- **Do not use** if the product has been contaminated or has come into contact with non-sterile surfaces
- **Always use** a clean applicator or gloves when handling to avoid contamination
- **It is important** that the bottle is not stored after opening.
The product must be used immediately and discarded after use to avoid the risk of contamination

Storage

- Store at room temperature
- Protect from direct sunlight and frost

Warnings

- For external use only
- Do not use if the packaging is damaged
- Avoid contact with eyes
- Discard if the entire contents are not used – the product cannot be stored after opening

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