

Activation of the BUSTER Gel Pack

1. Remove the BUSTER Gel Pack from its packaging
2. Submerge in luke warm water
3. Squeeze vigorously under water
4. Continue for about 3 minutes or until the cells are fully expanded

Alternatively the BUSTER Gel Pack can be placed in a washing machine on a rinse cycle. Multiple packs can be activated in this way.

After several uses the gel packs may need some reactivation. Should this be required, repeat steps 2-3.



Instructions for Use

Cold

1. Make sure the activated BUSTER Gel Pack is free of moisture.
2. Place flat in a freezer or the freezer compartment of a fridge. Leave for 24 hours or until the pack is sufficiently chilled.
3. Place the frozen/chilled gel pack into the special pockets in the BUSTER Kennel Mats. Alternatively wrap in a towel if not using with the kennel mats. Do not allow the animal to come into direct contact with the frozen gel packs.

Hot

1. Microwave on full power for 2 minutes.
2. Check the temperature of the BUSTER Gel Pack at various sites. If still too cold microwave for a further 30 seconds. If too hot allow to cool before use.
3. Place the BUSTER Gel Pack into the pockets in the BUSTER Kennel Mats. Alternatively wrap it in a towel if not using it with the BUSTER Kennel Mats. Do not allow the animal to come into direct contact with the BUSTER Gel Pack.

For veterinary use only



- 271720 - BUSTER ICU Kennel Mattress, with heating/cooling elements, S
- 271721 - BUSTER ICU Kennel Mattress, with heating/cooling elements, M
- 271722 - BUSTER ICU Kennel Mattress, with heating/cooling elements, M/L
- 271723 - BUSTER ICU Kennel Mattress, with heating/cooling elements, XL
- 271724 - BUSTER ICU gel pack for kennel mattress, 2/pk

ALWAYS CHECK THE TEMPERATURE OF THE BUSTER GEL PACKS BEFORE USE.
The gel packs are non toxic if consumed.