



BUSTER Foley Catheter, Silicone

2-way catheter in 100% silicone with 3-5 cc (ml) retention balloon and stylet.
Catheter has rounded tip and atraumatic side holes, and as silicone is very biocompatible, it is suitable for longer placement in urethra/bladder (up to 7 days).


- Silicone catheter with high biocompatibility
- Suitable for long term placement
- Easy-removable steel wire
- Colour-coded cap for easy size recognition
- 5/pk

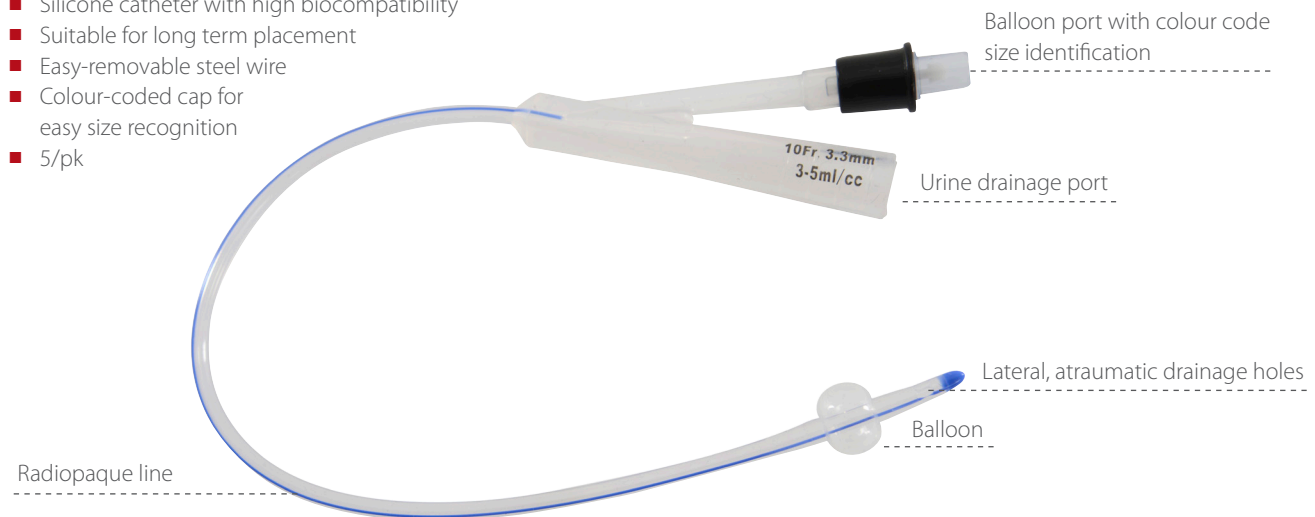


How to use...

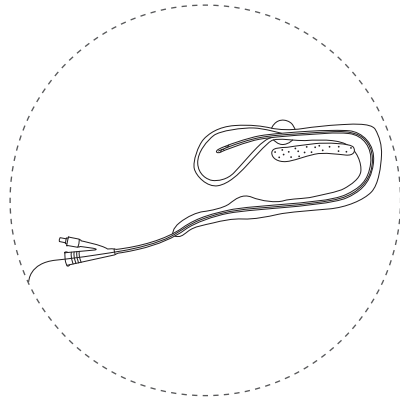

Insert the catheter through urethra into the bladder

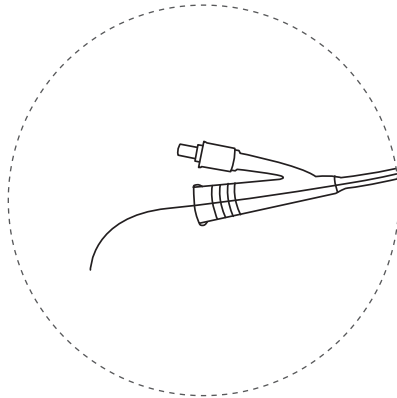

Gently remove the steel wire

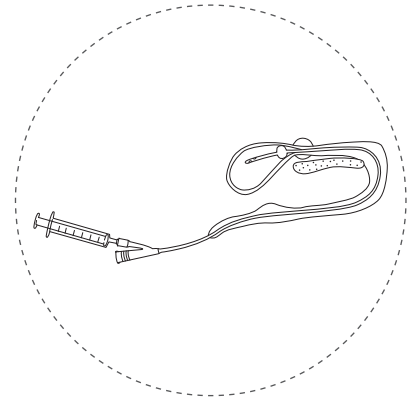

Fill the balloon with isotonic saline

Assortment

Cat. No	Size, Metric	Size, Imperial	Colour
273860	2 mm x 30 cm	6 Fr x 12 in	●
273861	2 mm x 55 cm	6 Fr x 22 in	●
273862	2.6 mm x 30 cm	8 Fr x 12 in	●
273863	2.6 mm x 55 cm	8 Fr x 22 in	●
273858	2.6 mm x 70 cm	8 Fr x 28 in	●
273864	3.3 mm x 30 cm	10 Fr x 12 in	●
273865	3.3 mm x 55 cm	10 Fr x 22 in	●
273859	3.3 mm x 70 cm	10 Fr x 28 in	●