

Assembly Instructions for ICU Cages



Remove the ICU cage from its carrying case



Undo the plastic clips on either side



Gently raise up the side of the cage



Lift up the top bars on one side



Gently slide back the spring loaded collar and align the bars. Gently release the spring loaded collar securing the bar.
Repeat on the other top bar



Insert the foam base. If using the gel packs (hot or cold) it is easier to put these in the foam base mat before putting the mat in the ICU cage



The unit is now ready for use. After use collapsing for storage is the reverse of these assembly instructions.
PLEASE ensure that the foam base is removed **BEFORE** the frame is collapsed.