

# KRUISE Chew-Stop

Discourage licking or biting and encourage healing.


Our best-selling Vet-Flex bandages are now available with a lick and chew deterrent!

Impregnated with a non-toxic bitter ingredient, Chew-Stop protects wounds after surgery or wound dressing change and allows for healing with less risk of interference from a curious pet.

Cat. No	Description
160675	KRUISE Chew-Stop, 5 cm x 4.5 m, 10/pk
160676	KRUISE Chew-Stop, 7.5 cm x 4.5 m, 10/pk
160677	KRUISE Chew-Stop, 10 cm x 4.5 m, 10/pk



**KRUISE VET-FLEX BANDAGE QUALITY  
- IN A REALLY BAD TASTE**

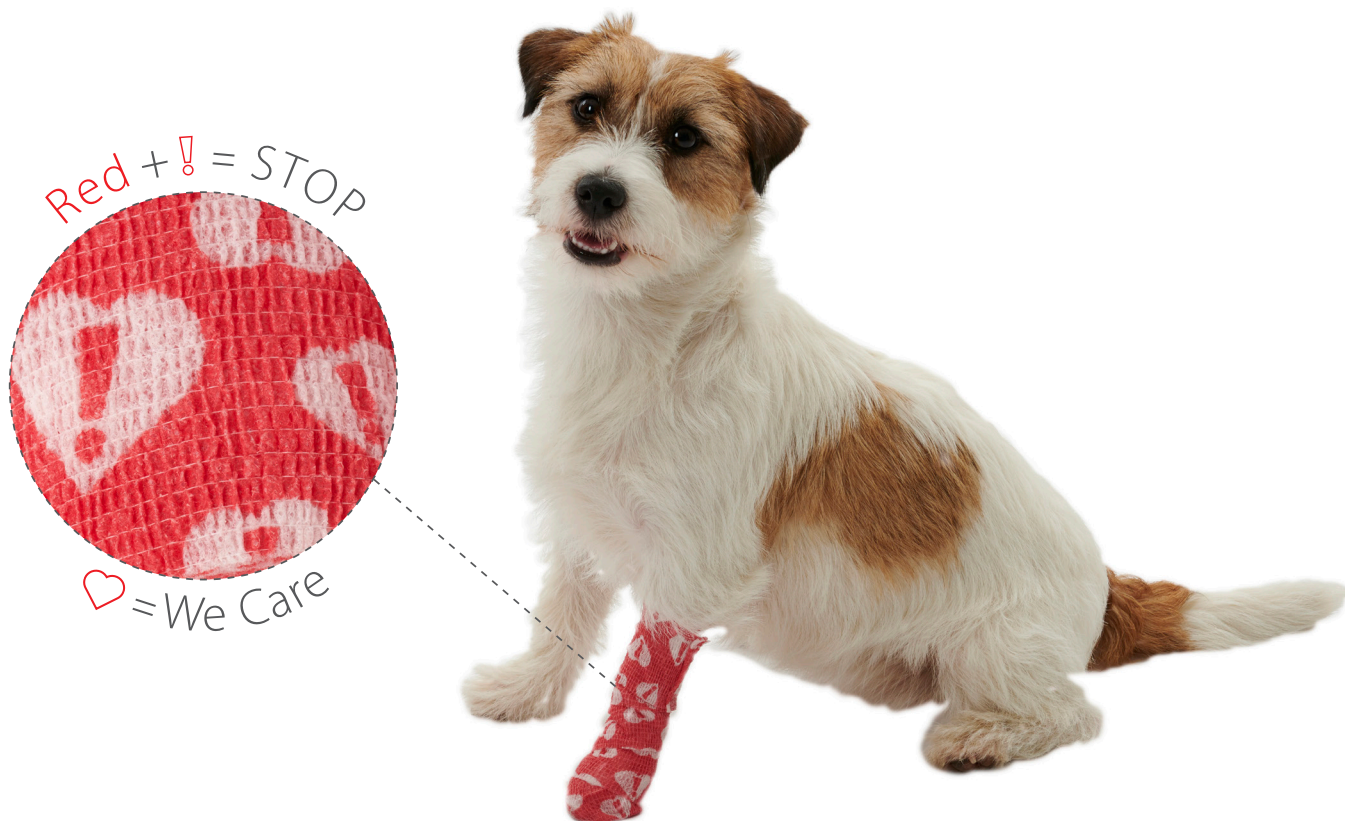

## Did you know?



- Licking a wound increases the risk of infection and tissue or suture damage, which may prolong healing
- It is a myth that dog saliva contains antibacterial properties
- You can use the BUSTER Bootie for additional wound protection on top of the KRUISE Chew-Stop bandage

**Application**

1. Fixate the end of the bandage and apply evenly over the injured body part
2. Stretch the bandage according to the level of compression needed
3. When application is done, tear the bandage without twisting it
4. Gently squeeze the layers together to enhance the cohesiveness

**Use for**

- Use KRUUSE Chew-Stop as a secondary/tertiary bandage on top of a primary/secondary dressing to prevent animals from licking or biting the bandage

**Removal**

- Unroll the bandage or cut gently with bandage scissors, being careful not to cut into the skin

**Do not use**

- If the patient or user is allergic to latex

For veterinary use only



## Let's wrap it up!

**Reliable and stable fixation**

The high weight of the material, 35 g/m<sup>2</sup>, makes the bandage strong and firm, yet comfortable. Provides excellent compression.

**Easy to tear**

No scissors needed! Simply pull in opposite directions to tear in a straight line.

**Skin-friendly**

KRUUSE Chew-Stop is coated with latex, which makes the bandage cohesive. It doesn't stick to surfaces like skin or hair and the porosity makes it breathable, hence it is skin-friendly.

**Very flexible**

KRUUSE Chew-Stop can be stretched to almost double the unrolled length! You can bandage all types of injuries on the animal's body and still control the compression.

**Excellent elasticity**

Spandex threads provide suitable shrinkage, which is necessary to give firm compression and fixation. With proper application the bandage will not constrict.

**Easy to remove**

The features of KRUUSE Chew-Stop cohesive bandages make them very easy to remove even without scissors.

**Hygienic**

The bandage is water-resistant, which helps prevent water from penetrating the layers. Each roll is individually wrapped in an easy-to-tear pouch.

**Good patient and user compliance**

The fact that the bandage is very light, does not constrict and is not sticky to touch makes it more comfortable for the animal and easy to work with for the vet and owner.



Easy to tear



Latex threads



Spandex threads