


KRUUSE Wound Care



Index

Moist Wound Healing	4
Manuka Honey Products	6
Hydro Gel	7
Primary and Secondary Layer Dressings	10
Othaematoma Pad	12
Cotton Products	13
Bandages	14
Immobilisation	16
Tail Splint	18
Flex Bandages	20
Vet-Plast	27
Protective Boots	28
Gauze and Nonwoven Products	30
Small Animal Wound Chart	36
Large Animal	38
Wound Flushing	40
Sprays	41
Monkey Splint	44
Body Bandages	44
Equine Masks	47
Equine Wound Chart	50

Moist Wound Healing

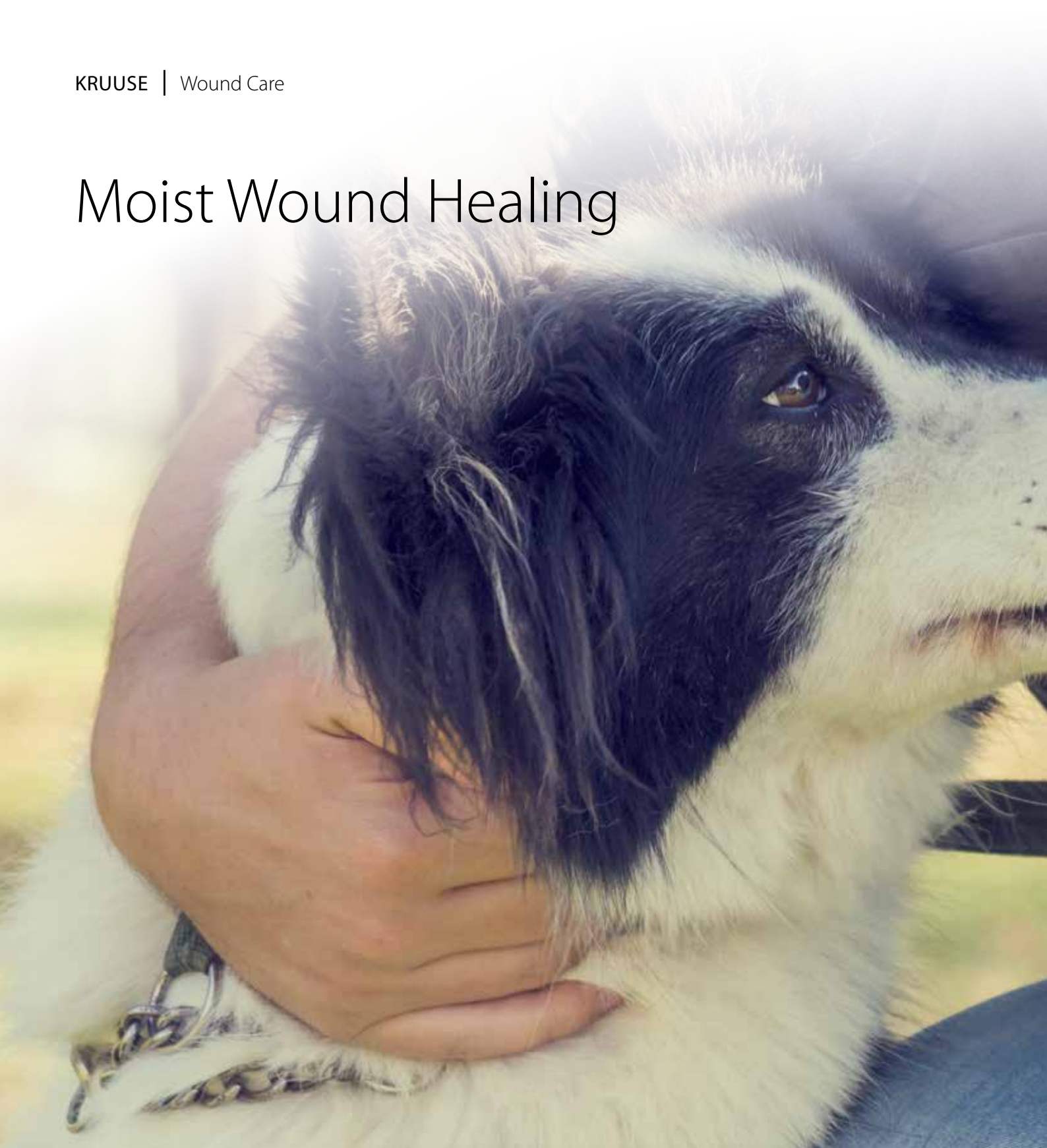

Moist wound healing ensures the quickest healing process, reduces pain, encourages vapor transmission, promotes autolytic debridement, and acts as a bacterial barrier. KRUUSE wound products are designed to positively influence the wound environment and to facilitate optimal wound healing

Faster wound healing

- 24 hours continuous wound healing as the wound is never allowed to dry out
 - Skin cells regenerate at approximately twice the speed in a moist environment
 - Contact between wound and wound fluid is maintained ensuring a normal healing balance
- The retentive dressing prevents infection from the outside providing faster healing
 - Excessive exudate is removed, leaving the wound bed moist and not wet



More comfort

- Keeping the nerve ends hydrated reduces pain
- Decreased pain when the dressing is removed
- Fewer dressing changes - each dressing lasts 3-5 days

Improved outcome

- Less and nicer scarring due to improved collagen fiber alignment
- Animal and owner compliance due to better healing process and less pain

Savings

- Reduced dressing and bandage consumption
- Decreased need for sedation

Manuka Magic

Manuka honey contains Methyl Glyoxal (MGO), which is unique since it continues to work as an antibacterial even after hydrogen peroxide wears out. Clean wounds are essential for faster wound healing.

The healing power of honey

The features of KRUUSE Manuka Honey support autolytic debridement and enhance moist wound healing in many ways.

Perfect acidic environment

The pH of honey (3.6-3.7) creates the desired acidic environment which has been proven to decrease bacterial growth, increase fibroblast activity, and increase oxygen release, all of which promote wound healing.

High osmolarity


Increased osmolarity draws fluid and lymph from the underlying tissues providing nourishment to the healing wound.

Glucose oxidase effect

Glucose oxidase produces hydrogen peroxide and gluconic acid, which provide the main antibacterial qualities of honey. The well-tolerated, low levels of hydrogen peroxide promote angiogenesis and fibroblast activity, enhancing oxygen delivery to the tissue.

Look out for this stamp!

Our new MGO 300+ stamp on our Manuka products is your guarantee for very high MGO activity and medical grade honey.

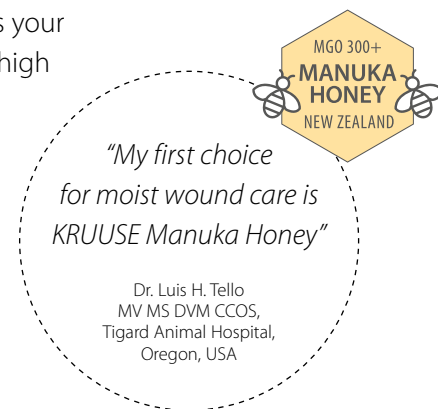

Cat. No	Description	Type	Size	Package
165010	KRUUSE Manuka G	Gel, sterile	15 g	10/pk
165011	KRUUSE Manuka G	Gel, sterile	50 g	6/pk

KRUUSE HydroGel

A clear amorphous hydrogel containing a Carboxymethyl cellulose polymer, propylene glycol and water. 0.04% Polyhexanide added.

Key Functions:

- Part of the wound preparation
- Provides a moist wound healing environment
- Promotes natural debridement through autolysis
- Loosens and absorbs slough and exudate
- Not runny
- Contains 0.04% Polyhexanide (PHMB) as a preservative



Stays in place.
Easy to apply.



Scan QR code
for more
information



Cat. No	Description	Type	Size	Package
165020	KRUUSE HydroGel w. Polyhexanide 0.04%	Sterile	15 g	10/pk

KRUUSE Manuka Honey ND non-adherent dressing

- For wounds with little exudate or in combination with KRUUSE Manuka gel
- Honey impregnated into acetate gauze



Cat. No	Description	Type	Size, Metric	Size, Imperial	Package
165000	KRUUSE Manuka ND	non-adherent dressing, sterile	5 x 5 cm	2" x 2"	10/pk
165001	KRUUSE Manuka ND	non-adherent dressing, sterile	10 x 12.5 cm	4" x 5"	10/pk
165002	KRUUSE Manuka ND	non-adherent dressing, sterile	10 x 100 cm	4" x 39"	1 roll



KRUUSE Manuka Honey AD absorbent dressing

- For wounds with moderate to heavy exudate
- Honey impregnated into a pad of super absorbent polymers



Cat. No	Description	Type	Size, Metric	Size, Imperial	Package
165005	KRUUSE Manuka AD	absorbent dressing, sterile	5 x 5 cm	2" x 2"	10/pk
165006	KRUUSE Manuka AD	absorbent dressing, sterile	10 x 12.5 cm	4" x 5"	10/pk
165007	KRUUSE Manuka AD	absorbent dressing, sterile	10 x 100 cm	4" x 39"	1 roll

KRUUSE Foam Dressing

A very good and comfortable primary or secondary wound dressing for absorption and maintaining a moist wound environment. This soft foam dressing conforms perfectly into the wound bed and helps excess exudate away from the wound.

- Maintains a moist wound healing environment
- Conforms to the wound bed
- Reduces pain
- Non-adherent
- Easy change of wound dressing



Cat. No	Description	Type	Size, Metric	Size, Imperial	Package
165030	KRUUSE Foam Dressing	Sterile	5 x 5 cm	2" x 2"	10/pk
165031	KRUUSE Foam Dressing	Sterile	10 x 10 cm	4" x 4"	10/pk
165032	KRUUSE Foam Dressing	Sterile	15 x 15 cm	6" x 6"	5/pk
165033	KRUUSE Foam Dressing	Sterile	10 x 20 cm	4" x 8"	10/pk



Instructions for use

Application

After wound bed preparation simply apply the dressing needed.

Evaluate the wound for amount of exudate and time to the next dressing change.

Use KRUUSE Manuka Honey as primary dressing alone or in combination with KRUUSE Hydro Gel.

Primary dressing suggestions

- KRUUSE Manuka Honey
- KRUUSE Lite-Absorb
- KRUUSE Super-Absorb

Secondary dressing suggestion

- KRUUSE Foam Dressing
- KRUUSE Lite-Absorb
- KRUUSE Super-Absorb

Tertiary bandage

- KRUUSE Vet-Plast
- KRUUSE Vet-Flex
- KRUUSE Soft-Flex bandages
- BUSTER Tubular Bandage

Removal

- Remove dressing and flush away the honey to evaluate the wound and e.g. do debridement.

KRUUSE Super-Absorb

Absorbent dressing with high absorbency properties for wounds with moderate to severe exudation. The excellent absorbent capacity reduces number of bandage changes and ensures better wound healing.

Application

The dressing works perfectly as primary or secondary dressing for managing wounds of different sizes, shapes, and amounts of exudate. KRUUSE Super-Absorb effectively contains and encapsulates exudate and cellular debris and thus prevents micro-organisms from contaminating the wound. Easily fixate the dressing with adhesive tape or a bandage. If the wound is highly infected, it is recommended to combine the dressing with systemic and/or topical antibiotic or antibacterial treatment.

Improved healing

The 3 dressing layers contribute to improved healing without causing maceration. The dressing protects the wound and absorbs wound exudate in a moist environment which promotes healing.

More comfort

Due to high absorption capacity and fluid retention, the dressing can stay on for days. The dressing will not stick to the wound which minimises pain when it is removed or changed.

Diversion layer


Quickly transfer the wound exudate to the absorbent layer

Super Absorbent Layer

Absorption capacity up to 140 g/100 cm²
It absorbs and encapsulate the exudate

Protective Back

Waterproof, bacteria-resistant, breathable wound protection

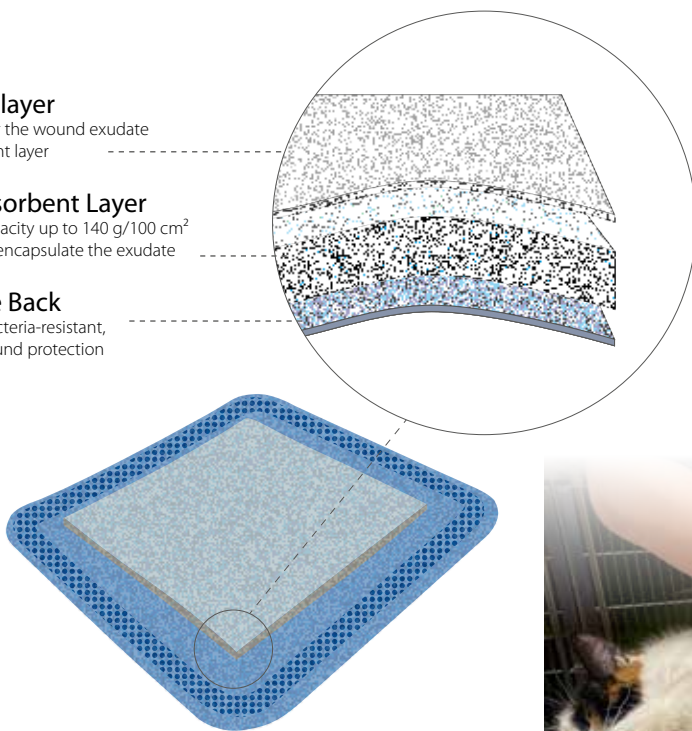

Scan QR code to watch the video

Cat. No	Description	Size, Metric	Size, Imperial	Package
160319	KRUUSE Super-Absorb	10 x 10 cm	4" x 4"	10/pk
160320	KRUUSE Super-Absorb	10 x 20 cm	4" x 8"	10/pk
160321	KRUUSE Super-Absorb	20 x 20 cm	8" x 8"	10/pk
160322	KRUUSE Super-Absorb	20 x 30 cm	8" x 12"	10/pk

KRUUSE Lite-Absorb

Absorbent dressing with light absorbency properties for use in mildly exudating wounds.

Application

Use as a primary layer dressing to cover wounds and absorb wound exudate or as a secondary layer to manage a variety of wound conditions.

Easily fixate the dressing with a gauze bandage or tape.


Ideal for use with KRUUSE Hydrogel or KRUUSE Manuka Honey which both contribute to keeping the wound moist.

Improved healing

The dressing is made of viscose and polyester and has a perforated PE (polyethylene) film on both sides reducing the risk of attachment to the wound bed. The tightly woven layers ensure removal of the bandage in one piece, without leaving lint in the wound.

More comfort

The cushioned inner layer protects the wound, and the non-adherent PE-film ensures decreased pain during dressing change and removal.

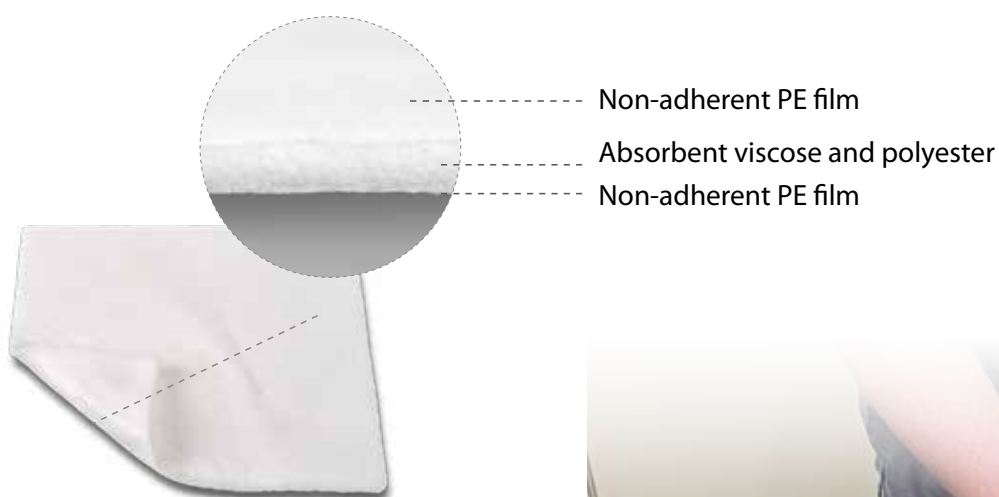


Scan QR code to watch the video

Cat. No	Description	Size, Metric	Size, Imperial	Package
160315	KRUUSE Lite-Absorb	5 x 5 cm	2" x 2"	25/pk
160316	KRUUSE Lite-Absorb	10 x 10 cm	4" x 4"	25/pk
160317	KRUUSE Lite-Absorb	10 x 20 cm	4" x 8"	25/pk

BUSTER Othaematoma

Compression pad for treatment of othaematoma. The pad provides compression and gives good ventilation. The BUSTER Othaematoma pad helps to re-attach the skin via suturing and compression.

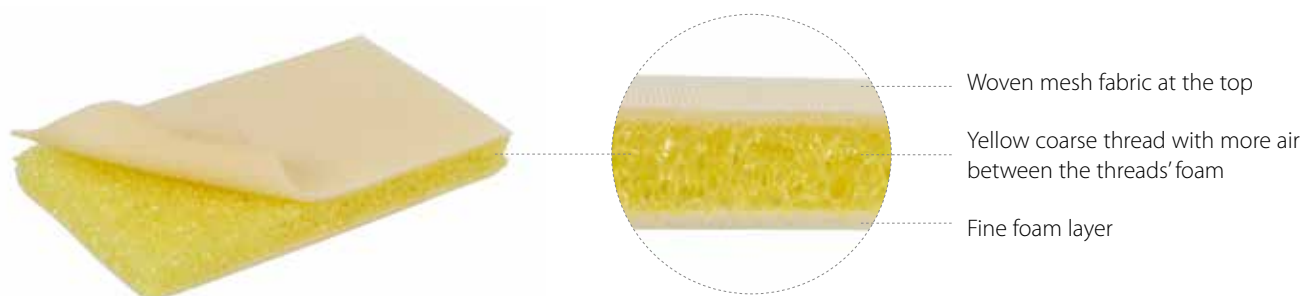

Cat. No	Description	Type	Size, Metric	Size, Imperial	Package
161640	BUSTER Othaematoma Pad	Sterile	12 x 8 cm	4 3/4" x 3"	10/pk

KRUUSE Steri-Protect

Sterile veterinary dressing with excellent absorbency and free from added chemicals.

Instructions for use

Hot dressing

- Boiled water is placed in a shallow container and cooled to 37°C
- Cut the dressing to the size and shape required and place in the water
- When saturated, remove from the water and squeeze out excess liquid
- Place over the affected area with the shiny side away from the skin
- Secure with a bandage

Cold dressing

- Soak the dressing in boiled water as described above, and let the dressing cool completely
- Apply to affected area

The dressing can also be chilled in the refrigerator and kept in the bag.

Dry dressing

- Apply to the affected area and secure with a bandage
- Avoid applying the bandage too tightly
- Always apply with shiny side away from the skin



Cat. No	Description	Type	Size, Metric	Size, Imperial	Package
160700	KRUUSE Steri-Protect, vet. poultice dressing	Display box	20 x 40 cm	8" x 15 3/4"	10/pk

KRUUSE Cotton Wool, Non-absorbent

Water-repellent cotton wool. Can be used for bandaging a paw to avoid moisture. Place the cotton wool between the foot pads on the paw to avoid a humid environment and fungal infection.



Cat. No	Description	Size, Metric	Size, Imperial	Package
160130	KRUUSE Cotton Wool, Non-absorbent	10½"	27 cm	10/pk

KRUUSE Cotton, Absorbent

Absorbent cotton wool for cleaning and drying. For example for cleaning ears.



Cat. No	Description	Package
160131	KRUUSE Cotton, Absorbent	150 g

KRUUSE Cotton

Soft and comfortable absorbent cotton wool to be used as padding under compression or plaster bandages to increase patient comfort and prevent pressure injuries.



Cat. No	Description	Type	Size, Metric	Size, Imperial	Package
160133	KRUUSE Cotton Roll	Absorbent	15 cm	6"	Roll of 300 g
160134	KRUUSE Cotton	Half Compressed	35 cm	14"	1000 g
160135	KRUUSE Cotton Wool	Absorbent	27 cm	10½"	500 g
161720	KRUUSE Cotton Wool	Absorbent			1000 g
161729	KRUUSE Absorbent Cotton Wool	Tightly Rolled	25 cm	10"	1000 g
161730	KRUUSE Absorbent Cotton Wool	Softly Rolled	35 cm	14"	1000 g

KRUUSE Cotton Balls

Absorbent cotton wool for cleaning, drying, wiping and protection where a sterile product is not required.



Cat. No	Description	Package
160132	KRUUSE Cotton Balls	100/pk

KRUUSE Soft-flex, Adhesive

KRUUSE Soft-Flex Adhesive is an adhesive elastic bandage with one side covered with porous foam, which allows the skin to breathe. Good for bandaging in difficult places. Do not apply directly to the wound.



Cat. No	Description	Type	Size, Metric	Size, Imperial	Package
160963	KRUUSE Soft-flex	Adhesive	5 cm x 2.5 m	2" x 8 1/4 ft	1/pk
160964	KRUUSE Soft-flex	Adhesive	10 cm x 2.5 m	4" x 8 1/4 ft	1/pk
160965	KRUUSE Soft-flex	Adhesive	15 cm x 2.5 m	6" x 8 1/4 ft	1/pk

KRUUSE Soft-flex, Cohesive

KRUUSE Soft-Flex Cohesive is a unique soft elastic bandage, elastic bandage, which sticks to itself and not to skin or fur.

Used for all kinds of wound dressings as a secondary layer. Used as support for plaster in connection with fractures, and for paw dressings.



Cat. No	Description	Type	Size, Metric	Size, Imperial	Package
160966	KRUUSE Soft-flex	Cohesive	5 cm x 2.5 m	2" x 8 1/4 ft	1/pk
160967	KRUUSE Soft-flex	Cohesive	10 cm x 2.5 m	4" x 8 1/4 ft	1/pk
160968	KRUUSE Soft-flex	Cohesive	15 cm x 2.5 m	6" x 8 1/4 ft	1/pk

KRUUSE Fixating Bandage

Soft and conforming elastic fixating bandage for keeping wound pads or padding in place. The breathable and skin-friendly fabric conforms to the body shape and provides extra patient comfort.



Cat. No	Description	Type	Size, Metric	Size, Imperial	Package
160350	KRUUSE Fixating Bandage	Elastic	6 cm x 4 m	2 2/5" x 13 ft	10/pk
160351	KRUUSE Fixating Bandage	Elastic	8 cm x 4 m	4" x 13 ft	10/pk
160352	KRUUSE Fixating Bandage	Elastic	10 cm x 4 m	3" x 13 ft	10/pk
160353	KRUUSE Fixating Bandage	Elastic	12 cm x 4 m	4 2/3" x 13 ft	10/pk

BUSTER Tubular Bandage

This bandage is very easy to change and provides air flow to aid healing. Custom fit for difficult-to-bandage areas (e.g. on the limbs). No tape is needed to secure the bandage. It provides a firm, yet comfortable pressure and does not unravel when trimmed or torn.



Scan QR code for more information



Cat. No	Description	Colour	Size, Metric	Size, Imperial
160480	BUSTER Tubular Stretch Bandage	White	1.25 cm x 6.5 m	1/2" x 21 1/4 ft
160481	BUSTER Tubular Stretch Bandage	White	2.5 cm x 6.5 m	1" x 21 1/4 ft
160482	BUSTER Tubular Stretch Bandage	White	3.8 cm x 6.5 m	1 1/2" x 21 1/2 ft
160484	BUSTER Tubular Stretch Bandage	White	7.5 cm x 6.5 m	3" x 21 1/2 ft
160485	BUSTER Tubular Stretch Bandage	White	8.8 cm x 6.5 m	3,5" x 21 1/2 ft

BUSTER OrthoVet Splint

Lower limb injuries in companion animals frequently need to be splinted or braced. The BUSTER OrthoVet splints are modelled after human splints, which have become increasingly popular for lower leg, ankle and foot injuries.

The brace is fixed using hook and loop straps. Available in various sizes for both front and hind limbs. The brace maintains the limb's natural walking position and allows the patient to bear weight on the injured limb, and it provides extra support for fractures.

It reduces the costs associated with bandaging, degloving injuries and casting. The lightweight design increases the patient's tolerance threshold with 3 wide hook and loop straps to efficiently fixate the brace in place. The brace for the hind limb reaches above the hock. The front limb brace reaches just below the elbow. Both braces extend underneath the foot.


Patient comfort is prioritised, since the brace allows free and effortless movement. It can be custom fitted to the individual patient by cutting or heating.



Cat. No	Description	Type	Size, Metric	Size, Imperial
272111	BUSTER OrthoVet Splint	Cat, hind limb	XS (12.50 cm)	5"
272112	BUSTER OrthoVet Splint	Dog, front limb	XS (13.75 cm)	5.5"
272106	BUSTER OrthoVet Splint	Dog, front limb	S (16.25 cm)	6.5"
272107	BUSTER OrthoVet Splint	Dog, front limb	M (18.75 cm)	7.5"
272108	BUSTER OrthoVet Splint	Dog, front limb	L (21.75 cm)	8.5"
272109	BUSTER OrthoVet Splint	Dog, front limb	XL (23.75 cm)	9.5"
272113	BUSTER OrthoVet Splint	Dog, hind limb	XS (17.50 cm)	7"
272114	BUSTER OrthoVet Splint	Dog, hind limb	S (20.0 cm)	8"
272115	BUSTER OrthoVet Splint	Dog, hind limb	M (22.5 cm)	9"
272116	BUSTER OrthoVet Splint	Dog, hind limb	L (27.5 cm)	11"
272117	BUSTER OrthoVet Splint	Dog, hind limb	XL (31.25 cm)	12"

BUSTER Soft Splint

Aluminium strip covered on both sides with foam padding. Easy to cut with ordinary scissors and can be formed into any shape. The soft splint is very flexible and radiolucent.

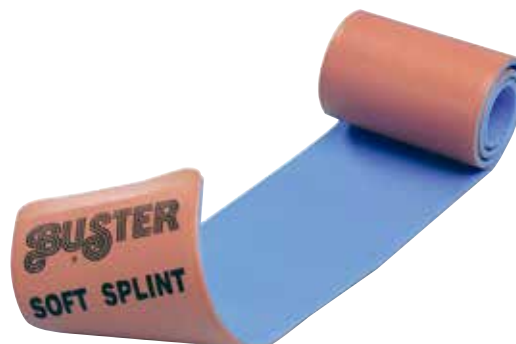

Cat. No	Description	Size, Metric	Size, Imperial	Package
272285	BUSTER Soft Splint	1 m	40"	roll

BUSTER Quicksplint

Clear plastic tarsal splint for dogs and cats. Specially designed for easy and accurate fitting. The splint supports the leg from either the lateral or medial side when held together by a soft adhesive bandage. The splints are perforated to accommodate drainage and ventilation. Recommended for a variety of conditions including fractures, soft tissue trauma, post surgical care, first aid etc. Supplied as a pair.



Cat. No	Description	Type	Size	Package
272287	BUSTER Quicksplint	Front limb	S	pair
272288	BUSTER Quicksplint	Front limb	M	pair
272289	BUSTER Quicksplint	Front limb	L	pair
272290	BUSTER Quicksplint	Hind limb	S	pair
272291	BUSTER Quicksplint	Hind limb	M	pair
272292	BUSTER Quicksplint	Hind limb	L	pair

BUSTER Plastic Leg Splints

Radiolucent disposable leg splints in green plastic with foam padding. Can be contoured at the distal end by applying moderate heat.



Cat. No	Description	Size, Metric	Size, Imperial
272260	BUSTER Plastic Leg Splint	S (2.5 x 20 cm)	1" x 8"
272270	BUSTER Plastic Leg Splint	M (3.8 x 30 cm)	1½" x 8"
272280	BUSTER Plastic Leg Splint	L (5.0 x 40 cm)	2" x 8"

BUSTER Carpal Splint

These splints are made of heavy-duty material and reinforced with longitudinal ribs. A recessed area reduces pressure on the carpal pad. A slight bend in the distal end follows the natural resting position. The splints are provided with open-pore foam for extra patient comfort.



Cat. No	Description	Size
272101	BUSTER Carpal Splint	XS
272102	BUSTER Carpal Splint	S
272103	BUSTER Carpal Splint	M
272104	BUSTER Carpal Splint	L
272105	BUSTER Carpal Splint	XL

Tail Splint

Lightweight and strong plastic tail splint for protecting the tip of the tail during healing. Attach with tape and position so that the end of the tail can be treated from the side.



Cat. No	Description	Size	Package
272200	Tail Splint	6 x S, 6 x M, 6 x L	Combi set
272201	Tail Splint	S	6/pk
272202	Tail Splint	M	6/pk
272203	Tail Splint	L	6/pk





Let's wrap it up!

KRUUSE Flex Bandages

Reliable and stable

The high weight of the material, 35 g/m², makes the bandage strong and firm, yet comfortable. Provides excellent compression.

Easy to tear

No scissors needed! Simply pull in opposite directions to tear in a straight line.

Skin-friendly

KRUUSE Fun-Flex is coated with latex, which makes the bandage cohesive. It doesn't stick to surfaces like skin or fur and the porosity makes it breathable.

Very flexible

KRUUSE Fun-Flex can be stretched to almost double the unrolled length! You can bandage all types of injuries on the patient's body and still control the compression.

Excellent elasticity

Spandex threads provide suitable shrinkage, which is necessary to give firm compression and fixation. With proper application the bandage will not constrict.

Easy to remove

The features of KRUUSE Fun-Flex cohesive bandages make them very easy to remove even without scissors.

Hygienic

The bandage is water-resistant, which helps prevent water from penetrating the layers. Each roll is individually wrapped in an easy-to-tear pouch.

Good patient and user compliance

The fact that the bandage is very light, does not constrict and is not sticky to touch makes it more comfortable for the patient and easy to work with for the vet and owner.



Easy to tear



Latex threads



Spandex threads




KRUUSE Fun-Flex Care, Inside-out

Some nurses and vets prefer to bandage inside-out. On this Fun-Flex Care version the pattern is printed on the inside of the bandage which leaves it visible once the bandage is applied.



Send your patients home with a caring message. Wish them good luck, tell them they are a star or find your own special messages in the 5 patterns.

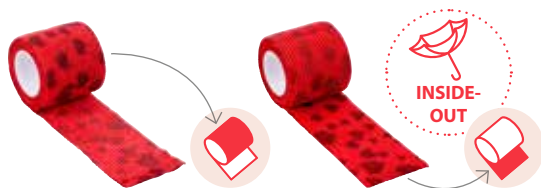

One way or the other

- Which way do you prefer to bandage?
- The print on the bandage determines which way you apply it on the patient
- KRUUSE Fun-Flex Care comes in both versions

Simply choose your direction!



Scan QR code
to watch
the video




Cat. No	Description	Colour	Size, Metric	Size, Imperial	Package
160715	KRUUSE Fun-Flex Care Inside-Out	2 of each pattern	5 cm x 4.5 m	2" x 15 ft	10/pk
160716	KRUUSE Fun-Flex Care Inside-Out	2 of each pattern	7.5 cm x 4.5 m	3" x 15 ft	10/pk
160717	KRUUSE Fun-Flex Care Inside-Out	2 of each pattern	10 cm x 4.5 m	4" x 15 ft	10/pk

KRUUSE Fun-Flex Care

We have put even more care into our well-known high quality KRUUSE Vet-Flex bandages.

Send your patients home with a caring message. Wish them good luck, tell them they are a star or find your own special messages in the Fun-Flex Care cohesive bandage range.

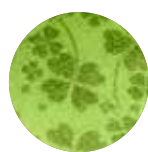

Lucky




Star



Love



Happy

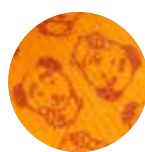

Pawfect



Cat. No	Description	Colour	Size, Metric	Size, Imperial	Package
164020	KRUUSE Fun-Flex Care	Mix of 5 different patterns	5 cm x 4.5 m	2" x 15 ft	10/pk
164022	KRUUSE Fun-Flex Care	Mix of 5 different patterns	7.5 cm x 4.5 m	3" x 15 ft	10/pk
164023	KRUUSE Fun-Flex Care	Mix of 5 different patterns	10 cm x 4.5 m	4" x 15 ft	10/pk



KRUUSE Chew-Stop

Discourage licking or biting and encourage healing

Our best-selling Vet-Flex bandages are available with a lick and chew deterrent!

Impregnated with a non-toxic bitter ingredient, Chew-Stop protects wounds after surgery or wound dressing change and allows for healing with less risk of interference from a curious pet.



Did you know?



- Licking a wound increases the risk of infection and tissue or suture damage, which may prolong healing
- It is a myth that dog saliva has antibacterial properties
- You can use the BUSTER Bootie for additional wound protection on top of the KRUUSE Chew-Stop bandage

Cat. No	Description	Colour	Size, Metric	Size, Imperial	Package
160675	KRUUSE Chew-Stop	Red	5 cm x 4.5 m	2" x 15 ft	10/pk
160676	KRUUSE Chew-Stop	Red	7.5 cm x 4.5 m	3" x 15 ft	10/pk
160677	KRUUSE Chew-Stop	Red	10 cm x 4.5 m	4" x 15 ft	10/pk

KRUUSE Fun-Flex Pet Bandage

Flexible cohesive lightweight bandage of superior quality which easily adjusts to even the most difficult body outlines. Easy to tear.




Cat. No	Description	Colour	Size, Metric	Size, Imperial	Package
164002	KRUUSE Fun-Flex Pet Bandage	Assorted	5 cm x 4.5 m	2" x 15 ft	10/pk
164004	KRUUSE Fun-Flex Pet Bandage	Assorted	7.5 cm x 4.5 m	3" x 15 ft	10/pk
164006	KRUUSE Fun-Flex Pet Bandage	Assorted	10 cm x 4.5 m	4" x 15 ft	10/pk

KRUUSE Fun-Flex Summer


See the joy in your client's face, when you bandage their pet with these happy patterns. We have mixed Summer icons with our well-known high-quality KRUUSE Vet-Flex for this unique bandage. 2 of each pattern.



Watermelon

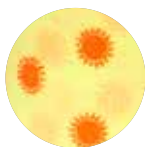

Sun

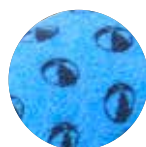

Ball



Palms



Ice cream cone

Cat. No	Description	Colour	Size, Metric	Size, Imperial	Package
164008	KRUUSE Fun-Flex Summer	Assorted	5 cm x 4.5 m	2" x 15 ft	10/pk
164009	KRUUSE Fun-Flex Summer	Assorted	7.5 cm x 4.5 m	3" x 15 ft	10/pk
164011	KRUUSE Fun-Flex Summer	Assorted	10 cm x 4.5 m	4" x 15 ft	10/pk


KRUUSE Fun-Flex Christmas

Wish your patients a Merry Christmas. The flexible cohesive bandages come in superior quality for fixation of dressings and controlled support of sprains and dislocations. Mix of 5 different patterns in a dispenser box of 10 strong, comfortable and user-friendly, easy to tear bandages.

Dispenser box ensures easy storage.



Christmas gift

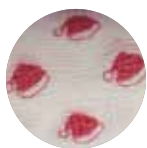


Santa hat



Snowman



Christmas tree

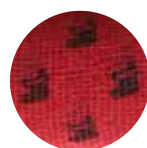

Christmas stocking

Cat. No	Description	Colour	Size, Metric	Size, Imperial	Package
164012	KRUUSE Fun-Flex Christmas	Assorted	5 cm x 4.5 m	2" x 15 ft	10/pk
164013	KRUUSE Fun-Flex Christmas	Assorted	7.5 cm x 4.5 m	3" x 15 ft	10/pk
164014	KRUUSE Fun-Flex Christmas	Assorted	10 cm x 4.5 m	4" x 15 ft	10/pk

KRUUSE Fun-Flex Bones

Superior quality of flexible cohesive bandage for fixation of dressings and controlled support of sprains and dislocations. Strong, comfortable, user-friendly and easy to tear. Mix of 5 different patterns.

Dispenser box for easy storage.



Cat. No	Description	Colour	Size, Metric	Size, Imperial	Package
164015	KRUUSE Fun-Flex Bones	Assorted	5 cm x 4.5 m	2" x 15 ft	10/pk
164016	KRUUSE Fun-Flex Bones	Assorted	7.5 cm x 4.5 m	3" x 15 ft	10/pk
164017	KRUUSE Fun-Flex Bones	Assorted	10 cm x 4.5 m	4" x 15 ft	10/pk

KRUUSE Vet-Flex

High quality flexible cohesive bandage. Strong, comfortable and user-friendly. The easy to tear Vet-Flex bandage has minimal shrinkage. Each roll is wrapped individually.



Cat. No	Description	Colour	Size, Metric	Size, Imperial	Package
160728	KRUUSE Vet-Flex	Red	2.5 cm x 4.5 m	1" x 15 ft	10/pk
160729	KRUUSE Vet-Flex	White	5 cm x 4.5 m	2" x 15 ft	10/pk
160730	KRUUSE Vet-Flex	Red	5 cm x 4.5 m	2" x 15 ft	10/pk
160731	KRUUSE Vet-Flex	Green	7.5 cm x 4.5 m	3" x 15 ft	10/pk
160732	KRUUSE Vet-Flex	White	7.5 cm x 4.5 m	3" x 15 ft	10/pk
160733	KRUUSE Vet-Flex	White	10 cm x 4.5 m	4" x 15 ft	10/pk
160734	KRUUSE Vet-Flex	Red	10 cm x 4.5 m	4" x 15 ft	10/pk
160735	KRUUSE Vet-Flex	Blue	10 cm x 4.5 m	4" x 15 ft	10/pk
160736	KRUUSE Vet-Flex	Green	10 cm x 4.5 m	4" x 15 ft	10/pk
160737	KRUUSE Vet-Flex	Yellow	10 cm x 4.5 m	4" x 15 ft	10/pk
160738	KRUUSE Vet-Flex	Black	10 cm x 4.5 m	4" x 15 ft	10/pk
160739	KRUUSE Vet-Flex	Assorted	10 cm x 4.5 m	4" x 15 ft	10/pk
160740	KRUUSE Vet-Flex	Assorted	7.5 cm x 4.5 m	3" x 15 ft	10/pk
160741	KRUUSE Vet-Flex	Assorted	5 cm x 4.5 m	2" x 15 ft	10/pk
160742	KRUUSE Vet-Flex	Red	15 cm x 15 m	6" x 15 ft	4/pk
160743	KRUUSE Vet-Flex	Blue	15 cm x 15 m	6" x 15 ft	4/pk
160744	KRUUSE Vet-Flex	Green	5 cm x 4.5 m	2" x 15 ft	10/pk
160745	KRUUSE Vet-Flex	Blue	5 cm x 4.5 m	2" x 15 ft	10/pk
160747	KRUUSE Vet-Flex	Red	7.5 cm x 4.5 m	3" x 15 ft	10/pk
160748	KRUUSE Vet-Flex	Blue	7.5 cm x 4.5 m	3" x 15 ft	10/pk

KRUUSE Vet-Plast Elastic Adhesive Bandage

Adhesive bandage made of white elastic cotton with a yellow application guide line in the center. Good ventilation, reverse winding and adhesive outer side to facilitate application. Can be used to fix a dressing in place and as support for sprains, joints, and fractures.



Cat. No	Description	Colour	Size, Metric	Size, Imperial	Package
161821	KRUUSE Vet Plast	White	5 cm x 4.5 m	2" x 15 ft	10/pk
161831	KRUUSE Vet Plast	White	7.5 cm x 4.5 m	3" x 15 ft	10/pk
161836	KRUUSE Vet Plast	White	10 cm x 4.5 m	4" x 15 ft	10/pk





BUSTER Bootie

Protective boot for active recovery

Veterinary outdoor cover boot made of waterproof and breathable nylon. Comfort and safety for patients when recovering from an injury or a surgical procedure.

- Waterproof and breathable nylon
- Seam-sealed and scuff guard
- Upper drawstring closure
- Adjustable elastic hook and loop straps for perfect fit
- Colour-marked sizes
- Machine-washable



- Upper drawstring closure
- Adjustable elastic hook and loop straps
- Waterproof/breathable nylon shell
- Machine-washable/hang dry
- Scuff guard




BUSTER Bootie,
Hard Sole



BUSTER Bootie,
Soft Sole

Cat. No Soft Sole	Cat. No Hard Sole	Size	Colour	Height	Top circumference	Sole circumference	
161678	161669	XS	Yellow	15.5 cm / 6"	20 cm / 8"	18.5 cm / 7"	
161679	161670	XS 2	Pink	21 cm / 8"	28 cm / 11"	22 cm / 9"	
161680	161671	Short	Orange	15 cm / 6"	26 cm / 10"	25.5 cm / 10"	
161681	161672	S	Cyan	27 cm / 11"	30.5 cm / 12"	26.5 cm / 10"	
161682	161673	M	Red	33 cm / 13"	38 cm / 15"	33 cm / 13"	
161683	161674	L	Green	46 cm / 18"	44.5 cm / 18"	36.5 cm / 14"	
161684	161675	XL	Purple	58 cm / 23"	53.5 cm / 21"	40.5 cm / 16"	
	161676	Robert Jones	Brown	53.5 cm / 21"	56 cm / 22"	44.5 cm / 18"	
161685	161677	BUSTER Bootie, Starter Pack XS, SH, S, M, L					

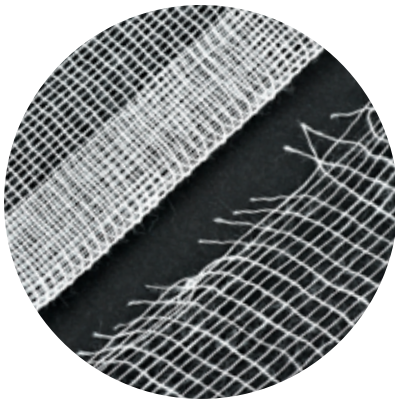

There is a clear difference between folded and unfolded edges. Loose lint are left behind in the wound and can cause infection/inflammation and prolong wound healing



Gauze balls in 4 regular sizes



Gauze range in different sizes and layers depending on usage/procedure




Raw cotton is removed from the seeds and leaves. It contains wax, proteins and minerals and needs to be processed in order to be absorbent




Gauze balls are made by turning edges of gauze inside forming a compact ball. KRUUSE balls are folded without the fixating silicone ring to avoid plastic waste




Cotton sample gently pulled apart for visual inspection of fiber length and impurities

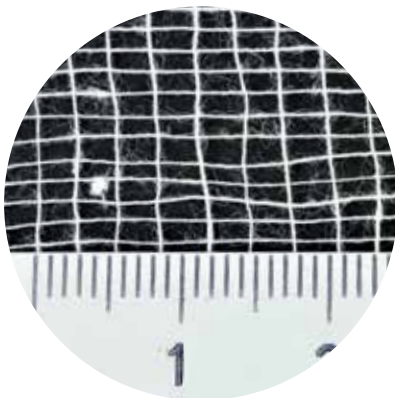

8 thread gauze has less threads per square inch, meaning it has less material and is less absorbent

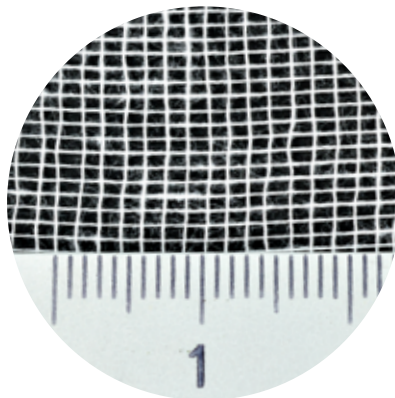

There is more material in gauze with 17 threads per square inch, meaning it offers higher absorption

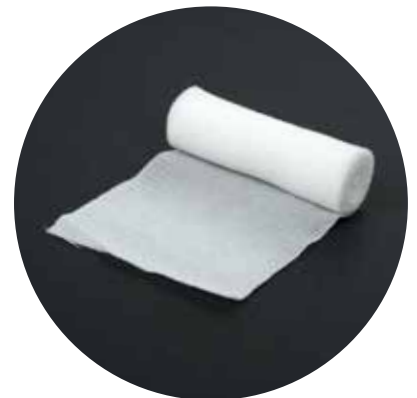

Very soft and conformable fixating bandage



Gauze life from seed to clinic

Gauze is made from cotton. Each cotton flower can contain up to 16,000 fibers. Each fiber grows in 50 days developing a cell consisting of mainly cellulose. The length of each fiber is dependent on the right growing conditions and can be as long as 3.6 cm with only 25 µm width and is barely visible to the human eye.

Quality matters

Long fibers can wrap better around each other and make stronger, softer and better fabrics, whereas shorter fibers tend to fray more, creating a less firm fabric that may leave loose fibers in the environment and the wound.

Cotton gauze emerges

After harvesting, the cotton flowers are brought to the factory, where the cotton is separated from the seed and the residue from the plant. Then, it is spun into a thread and woven into fabric creating the cotton gauze material.



Better absorption

After weaving, the material is treated through a scouring and bleaching process in order to be able to absorb liquid. Raw cotton contains natural wax, proteins and minerals, which will interfere with absorption in the finished product if not removed.

Know the differences

- to get best-in-class quality

- **Thread quality/thickness:** The more material, the better absorption
- **Longer fibers** equal less lint and **better quality**
- **Threads per sq/inch:** Higher thread count means more material and better absorption
- **Folded or unfolded swabs:** Folded swabs avoid loose threads, which can prolong healing time
- **Ply in swab:** The more layers the more material and better absorption



What is nonwoven?

“Nonwoven” is an expression that covers fabric-like material made from fibers bonded together by chemical, mechanical, heat or solvent treatment.

Nonwoven fabrics are among the most versatile materials on the market today and are also used for materials in your home like reusable shopping bags, furniture and even clothes. In the medical world, nonwoven material is widely used in gowns, masks, operating covers and swabs.

Nonwoven products for medical use

The combination of materials in nonwoven products for medical use usually contains polyester and viscose in a blend. Polyester is originally derived from petroleum and is therefore a chemical water-resistant ingredient, which binds the material and ensures stability. Viscose is made from wood pulp, which contains cellulose and is by nature a very absorbent material.



A dynamic, value-added alternative


Nonwoven fabrics are a dynamic, value-added alternative to more traditional materials. With nonwoven technology, fabrics can be designed to take on an array of characteristics that other materials simply cannot match:

- Better absorption per gram weight than competing fabrics - 40 g/m² provides excellent absorbency
- Very soft for sensitive and damaged skin
- High level of uniformity
- Lint free, leaves no residue in the wound
- High tear and tensile strength
- Very economical choice


The nonwoven material in the KRUUSE Nonwoven Swabs is composed of 50% polyester and 50% viscose. In conjunction with the high weight quality of 40 g/m² it provides you a versatile product with higher absorbency than regular 30 g/m² material.



KRUUSE Nonwoven Swabs in three sizes and traditional 4-layer

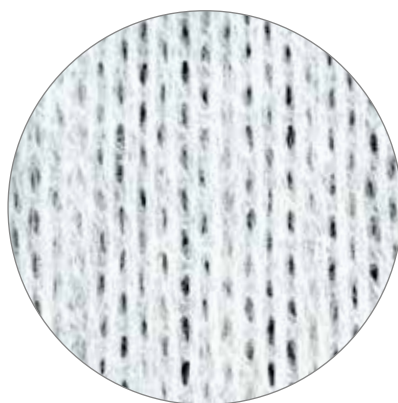

KRUUSE 40 g/m² material is soft when dry and wet and is appropriate for treating fragile and sore areas

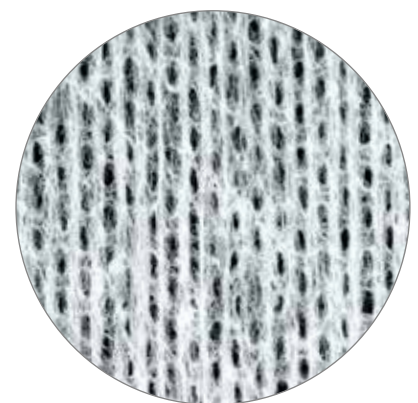

This 30 g/m² close up illustrates the difference in material and thus absorbency capacity compared to 40 g/m²

What is the difference between gauze and nonwoven materials?

Cotton gauze is a natural material, whereas nonwoven products are fabric-like materials made from fibers. See the tables below for comparisons of the two types of material and how you can best benefit from using them in your daily procedures.

Material comparison

Description	Gauze	Nonwoven
Materials	100% cotton	50 % viscose 50 % polyester
Natural material	Yes	Partly
Bleached	Yes	No
Lint residue	Medium risk	No risk
Soft in dry condition	Soft	Very soft
Soft in wet condition	Not as soft	Soft
Can be sterilised	Yes	Yes
Form stability in use	Very stable	Stable
Air permeability	High	Medium
Absorption capacity	Medium	High
Absorption speed	Medium	High
Fluid retention	High	Medium
Value for money	Good	Very good
Environmental waste impact	Poor	Good



Recommendation for use?

Description	Gauze	Nonwoven
Preparation before procedures/cleaning after surgery	✓✓✓	✓✓✓
For absorption during operations/procedures	✓✓✓	✓✓✓
Hemostasis	✓✓✓	✓✓✓
Cleaning eyes/ears/tender/sore areas of the body	✓	✓✓✓
Cleaning wound/changing dressing/peri wound	✓	✓✓✓
Disinfection before IV/blood sample/injection	✓	✓✓✓
After removal of IV/blood sample	✓✓	✓✓✓
Drainage	✓✓✓	✓✓✓
Dental cleaning/procedures	✓✓✓	✓✓✓

KRUUSE Nonwoven Swabs

Nonwoven swabs are made of a soft and comfortable material, making them very versatile for use in a multitude of procedures. Even in wet condition, the swabs remain soft, making them suitable for purposes such as washing a patient's sore ears or damaged skin. The composition of the material and the high weight quality of 40 g/m² ensures a high absorbency and softness.



Cat. No	Description	Size, Metric	Size, Imperial	Ply	Weight g/m ²	Package
160300	KRUUSE Nonwoven Swabs	5 x 5 cm	2" x 2"	4	40	100/pk
160301	KRUUSE Nonwoven Swabs	7.5 x 7.5 cm	3" x 3"	4	40	100/pk
160302	KRUUSE Nonwoven Swabs	10 x 10 cm	4" x 4"	4	40	100/pk

KRUUSE Gauze Swabs

For sterile or non-sterile use, KRUUSE Gauze Swabs come in various sizes and layers with a high thread count and folded edges ensuring high absorption and decreased risk of infection.



Faster healing

The folded edges in KRUUSE Gauze Swabs leave less debris and lint residue in the wound during procedures. Use the gauze swab as it is and do not cut the edges, as doing so might leave lint residue and prolong wound healing or cause infection/inflammation.


High quality equals high absorption

KRUUSE Gauze Swabs contain 17 thread quality because a high thread count equals more material and hence a higher absorption capacity, which also means less gauze is needed per procedure.

Cat. No	Description	Size, Metric	Size, Imperial	Ply	Thread	Package
160120	KRUUSE Gauze Swabs	5 x 5 cm	2" x 2"	8	17	100/pk
160121	KRUUSE Gauze Swabs	7.5 x 7.5 cm	3" x 3"	8	17	100/pk
160123	KRUUSE Gauze Swabs	10 x 10 cm	4" x 4"	8	17	100/pk
160184	KRUUSE Gauze Swabs	10 x 20 cm	4" x 8"	8	17	100/pk
160185	KRUUSE Gauze Swabs	5 x 5 cm	2" x 2"	12	17	100/pk
160186	KRUUSE Gauze Swabs	7.5 x 7.5 cm	3" x 3"	12	17	100/pk
160187	KRUUSE Gauze Swabs	10 x 10 cm	4" x 4"	12	17	100/pk
160188	KRUUSE Gauze Swabs	10 x 20 cm	4" x 8"	12	17	100/pk
160125	KRUUSE Gauze Swabs	10 x 10 cm	4" x 4"	16	17	100/pk
160335	KRUUSE Gauze Swabs, Sterile	5 x 5 cm	2" x 2"	12	17	5 x 20/pk
160336	KRUUSE Gauze Swabs, Sterile	7.5 x 7.5 cm	3" x 3"	12	17	5 x 20/pk
160337	KRUUSE Gauze Swabs, Sterile	10 x 10 cm	4" x 4"	12	17	5 x 20/pk
160338	KRUUSE Gauze Swabs, Sterile	10 x 20 cm	4" x 8"	12	17	5 x 20/pk

KRUUSE Gauze Swabs with X-ray thread

The sterile X-ray swabs with folded edges contain a special blue barium sulfate thread, which is radio-opaque. If, by accident a swab is missing after surgery, it is easy to detect the swab on radiographs without having to re-open the patient.

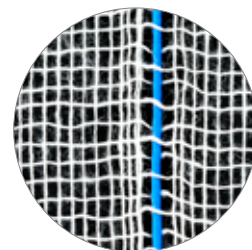

Radiopaque barium thread for easy recognition, if a swab is accidentally left in the operation site

Cat. No	Description	Size, Metric	Size, Imperial	Ply	Thread	Package
160340	KRUUSE Gauze Swabs , X-ray, Sterile	5 x 5 cm	2" x 2"	12	17	10 x 10/pk
160341	KRUUSE Gauze Swabs , X-ray, Sterile	7.5 x 7.5 cm	3" x 3"	12	17	10 x 10/pk
160342	KRUUSE Gauze Swabs , X-ray, Sterile	10 x 10 cm	4" x 4"	12	17	10 x 10/pk

KRUUSE Gauze Balls

High quality 17 threads per square inch gauze balls are available in different sizes for various procedures. The soft and breathable high absorbency KRUUSE Gauze Balls are manufactured without a silicone ring, to reduce the environmental impact.

Creating your own gauze balls at the clinic is not cost effective or safe. It can potentially generate too much lint residue, leading to prolonged wound healing and infection.



KRUUSE cotton ball folded without a silicone ring - ready for use in numerous procedures




Gauze ball with silicone ring





Cat. No	Description	Ø cm	Size, Metric non folded	Size, Imperial	Thread	Package
160345	KRUUSE Gauze Balls	2	15 x 15 cm	6" x 6"	17	250/pk
160346	KRUUSE Gauze Balls	3	20 x 20 cm	8" x 8"	17	250/pk
160347	KRUUSE Gauze Balls	4	25 x 25 cm	10" x 10"	17	250/pk
160348	KRUUSE Gauze Balls	5	30 x 30 cm	12" x 12"	17	250/pk

KRUUSE Small Animal Wound Chart

Dry 


Primary 

Secondary 

Tertiary 



- ▶ **Acute/traumatic wound**
- Evaluate the wound
- Clean the wound using KRUUSE wound lavage system
- Dress and bandage according to evaluation

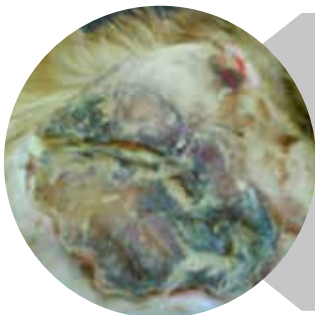


- ▶ **Necrotic**
- Debride the necrotic tissue and lavage
- Protect wound margins and surrounding skin from exudate

KRUUSE Manuka G
KRUUSE Manuka ND
KRUUSE HydroGel
KRUUSE Lite-Absorb

KRUUSE Foam
KRUUSE
Lite-Absorb

KRUUSE Vet-Plast
KRUUSE Flex Bandages
KRUUSE Soft-Flex
KRUUSE Tubular Bandage

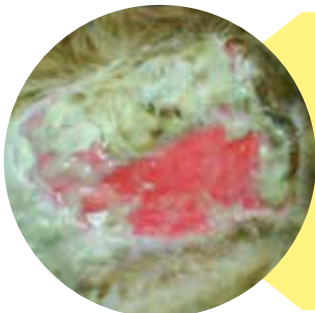

- ▶ **Sloughy**
- Debride devitalised tissue and lavage
- Protect wound margins and surrounding skin from exudate

KRUUSE Manuka G
KRUUSE Manuka ND
KRUUSE HydroGel
KRUUSE Lite-Absorb

KRUUSE Foam
KRUUSE
Lite-Absorb

KRUUSE Vet-Plast
KRUUSE Flex Bandages
KRUUSE Soft-Flex
KRUUSE Tubular Bandage



- ▶ **Granulating**
- Maintain a moist environment and protect new healthy granulation from damage between dressing changes

KRUUSE HydroGel
KRUUSE Lite-Absorb

KRUUSE Foam
KRUUSE
Lite-Absorb

KRUUSE Vet-Plast
KRUUSE Flex Bandages
KRUUSE Soft-Flex
KRUUSE Tubular Bandage



- ▶ **Epithelialising**
- Maintain a clean environment and protect from unnecessary trauma

KRUUSE HydroGel
KRUUSE Lite-Absorb

KRUUSE Foam
KRUUSE
Lite-Absorb


KRUUSE Flex Bandages
KRUUSE Soft-Flex
KRUUSE Tubular Bandage


Moderate exudate




Heavy exudate





Primary 

Secondary 

Tertiary 

Primary 

Secondary 

Tertiary 

--	--	--

--	--	--

KRUUSE Manuka G KRUUSE Manuka AD KRUUSE HydroGel KRUUSE Lite-Absorb KRUUSE Super-Absorb	KRUUSE Foam KRUUSE Lite-Absorb KRUUSE Super-Absorb	KRUUSE Vet-Plast KRUUSE Flex Bandages KRUUSE Soft-Flex KRUUSE Tubular Bandage
--	--	--

KRUUSE Manuka G KRUUSE Manuka AD KRUUSE Super-Absorb	KRUUSE Foam KRUUSE Super-Absorb	KRUUSE Vet-Plast KRUUSE Flex Bandages KRUUSE Soft-Flex KRUUSE Tubular Bandage
---	---------------------------------------	--

KRUUSE Manuka G KRUUSE Manuka AD KRUUSE HydroGel KRUUSE Lite-Absorb KRUUSE Super-Absorb	KRUUSE Foam KRUUSE Lite-Absorb KRUUSE Super-Absorb	KRUUSE Vet-Plast KRUUSE Flex Bandages KRUUSE Soft-Flex KRUUSE Tubular Bandage
--	--	--

KRUUSE Manuka G KRUUSE Manuka AD KRUUSE Super-Absorb	KRUUSE Foam KRUUSE Super-Absorb	KRUUSE Vet-Plast KRUUSE Flex Bandages KRUUSE Soft-Flex KRUUSE Tubular Bandage
---	---------------------------------------	--

KRUUSE Manuka G KRUUSE Manuka ND KRUUSE HydroGel KRUUSE Lite-Absorb KRUUSE Super-Absorb	KRUUSE Foam KRUUSE Lite-Absorb KRUUSE Super-Absorb	KRUUSE Vet-Plast KRUUSE Flex Bandages KRUUSE Soft-Flex KRUUSE Tubular Bandage
--	--	--

--	--	--

--	--	--

--	--	--





KRUUSE Wound Flushing Unit

KRUUSE Wound Lavage System facilitates optimal wound care. Effective wound irrigation is ensured by the use of 3 litres of body temperature isotonic fluid delivered at the correct pressure.

KRUUSE is the first company to develop a dedicated veterinary wound flushing unit with a documented and stabilised pressure of 0.82 bar (12 psi). The flushing unit delivers 3 litres of fluid in less than 5 minutes.



Scan QR code for more information



Cat. No	Description
165050	KRUUSE Wound Flushing Unit

KRUUSE Wound Flushing Solution

KRUUSE Wound Flushing Solution sachet contains the correct amount of electrolytes to add to the Wound Flushing Unit to produce 3 litres of body temperature physiological saline solution. The sachet also includes 0.04% Polyhexanide (PHMB) as a preservative. This solution is ideal for professional wound lavage.

One sachet contains 30,051 g powder consisting of:

- Sodium: 8.964 g
- Chloride: 11.591 g
- Lactate: 7.482 g
- Potassium: 0.470 g
- Calcium: 0.181 g
- PHMP: 1.200 g
- Water: 0.163 g



Cat. No	Description	Package
165051	KRUUSE Wound Flushing Solution	10 sachets



KRUUSE Wound Plast Spray

Sterile wound plaster spray. Transparent, water-resistant protective film without tar. Environmentally friendly. No CFCs.



Cat. No	Description	Size
161010	KRUUSE Wound Plast Spray	200 ml

KRUUSE Tar Plast Spray

KRUUSE Wound Plast Spray with tar. Sterile wound plaster spray. Transparent, water-resistant protective film with tar. Fly-repelling and environmentally friendly. No CFCs.



Cat. No	Description	Size
161020	KRUUSE Tar Plast Spray	200 ml

KRUUSE Aluminium Wound Spray

Astringent and antibacterial spray which promotes wound healing. Acts as a wound plaster allowing the wound to breathe while keeping it clean.



Cat. No	Description	Size
161035	KRUUSE Aluminium Wound Spray	200 ml

KRUUSE 0-bite Spray

The 0-bite pig perfume is very effective against tail bites and pig fights. KRUUSE 0-bite Spray can also be used for poultry.



Cat. No	Description	Size
161072	KRUUSE 0-bite Spray	400 ml

KRUUSE Blue-Spray

A disinfectant spray for bite wounds, cuts, and other wounds. Can be used for horses, dogs, cats, pigs, and cattle.



Cat. No	Description	Size
161080	KRUUSE Blue-Spray, Aerosol	200 ml
161081	KRUUSE Blue-Spray, Aerosol	400 ml

BOVIVET Ointment Spray

Zinc oxide ointment spray. Protects the skin against sweat, urine, water and other harmful substances. Environmentally friendly. No CFCs.



Cat. No	Description	Type	Size
161100	BOVIVET Ointment Spray	Skin Protection	200 ml

KRUUSE Teat Bandage

Unique polyester cohesive bandage. The allergy-tested teat bandage protects wounds against dirt. It is easy to mold and apply and can be applied to wet teats.



Cat. No	Description	Size, Metric	Size, Imperial
160954	KRUUSE Teat Bandage	6.0 cm x 4.5 m	2 1/4" x 15 ft

KRUUSE EquiPadding

High quality gauze-covered medical absorbent cotton wool for protection and wound care. Medical bleached absorbent cotton wool with fiber length 2-2.5 cm covered with gauze threads 18 x 26 per inch².

Gentle support

- Lower limb protection
- Support for tendons and joints

Good heat insulation

- Keeps tendons and muscles warm

Wound padding

- Gives superior protection to wounds whether they are dry, discharging or infected
- Standard wound care and management principles should always be applied
- Can be sterilised

Easy to use

- Can be cut into the exact size and shape required
- Secure with a support bandage or an elastic dressing
- Avoid overlapping as this could result in pressure points
- Where pressure points such as the knee are at risk a "comfort" hole should be made



Cat. No	Description	Size, Metric	Size, Imperial
165070	KRUUSE EquiPadding	15 cm	6"
165071	KRUUSE EquiPadding	30 cm	12"
165072	KRUUSE EquiPadding	45 cm	18"

EQUIVET Monkey Splint

The splint is made of 6 mm ABS plastic, conferring great strength and providing excellent support for distal limb injuries. The lower portion of the splint includes a block wedge designed to elevate and support the back of the hoof. In conjunction with the moulded dorsal portion of the splint, the wedge removes strain from the digital flexor tendons and immobilises completely the distal joints of the limb.

Suitable for treatment of severe tendon injuries, lacerations, and the immobilisation of distal limb fractures. The splint offers rapid and efficient support for transportation to the hospital or clinic. The material is fully radiolucent and is unaffected by water, blood and disinfectant solutions.

The Monkey Splint is ready for use incl. bandage kit (160570 contains a compress and gauze bandage).



Cat. No	Description	Type	Size, Metric	Size, Imperial
160555	EQUIVET Monkey Splint		L (17 cm)	6 3/4"
160560	EQUIVET Monkey Splint		M (14 cm) (Thoroughbred)	5.5"
160561	EQUIVET Monkey Splint		S (12 cm) (Foal)	4 3/4"
160570	EQUIVET Monkey Splint	Bandage Kit		

KRUUSE Recovery Bandage

3-dimensional elastic wrap which ends in a wide foam part covered in velour adhesive to Trihook* (*Extra strong adhesive strip). Interior pocket lined with a monofilm pad 1 cm thick acts as an air cushion covering the lower abdomen, making it easier for the horse to stand up. Useful for full protection in case of violent recovery and maximum immediate post-operative protection.



Cat. No	Description	Size
165200	KRUUSE Foal Recovery Bandage	S
165201	KRUUSE Foal Recovery Bandage	M
165202	KRUUSE Foal Recovery Bandage	L
165203	KRUUSE Equine Recovery Bandage	S
165204	KRUUSE Equine Recovery Bandage	M
165205	KRUUSE Equine Recovery Bandage	L

KRUUSE Post Colic Surgery Kits

Apply the postoperative bandage on the patient in less than three minutes. The outside is made of foam covered in velour adhesive to Trihook* (*Extra strong adhesive strip). The inside is made of 100% skin-friendly cotton. The bandage is breathable and the finishing has elastic ribbon and hence no friction points.

The kit includes:


- 1 abdominal wrap with a removable washable pocket (The mesh can be slit to replace absorbent drainage pad)
- 2 replacement interior pockets
- 3-4 dorsal straps (depending on the size)
- 1 chest strap



Cat. No	Description	Size
165206	KRUUSE Foal Post Colic Surgery Kit	S
165207	KRUUSE Foal Post Colic Surgery Kit	M
165208	KRUUSE Foal Post Colic Surgery Kit	L
165209	KRUUSE Equine Post Colic Surgery Kit	S
165210	KRUUSE Equine Post Colic Surgery Kit	M
165211	KRUUSE Equine Post Colic Surgery Kit	L
165323	KRUUSE Equine Chest Strap (165209)	S
165324	KRUUSE Equine Chest Strap (165210)	M
165325	KRUUSE Equine Chest Strap (165211)	L
165303	KRUUSE Foal Pockets 12 units (165206)	S
165305	KRUUSE Foal Pockets 12 units (165208)	L
165326	KRUUSE Equine Pockets 12 units (165209)	S
165327	KRUUSE Equine Pockets 12 units (165210)	M
165328	KRUUSE Equine Pockets 12 units (165211)	L

KRUUSE Scrotum Straps


This strap is an addition to the Post Colic Surgery Kit and is only to be used for male horses. The strap has three adjustment points to adjust the small hollow in front of the penis to prevent the horse from urinating inside the abdominal wrap. Fit the strap ventrally in front of the penis and fasten on the dorsal strap.

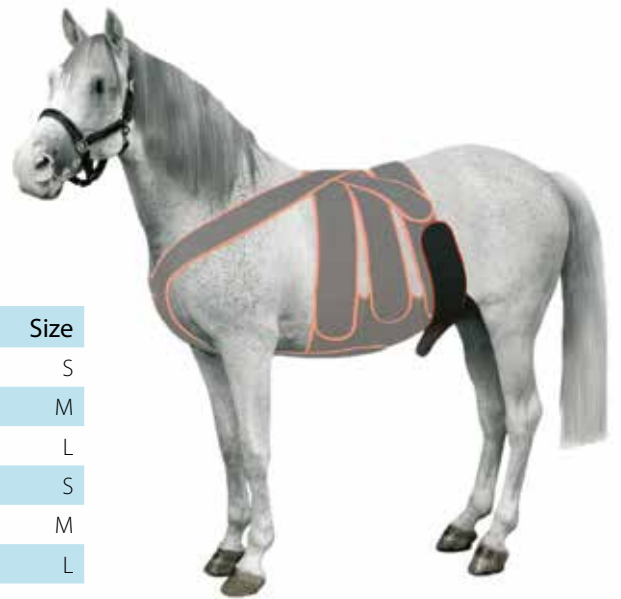

Cat. No	Description	Size
165300	KRUUSE Foal Scrotum Strap (165206)	S
165301	KRUUSE Foal Scrotum Strap (165207)	M
165302	KRUUSE Foal Scrotum Strap (165208)	L
165320	KRUUSE Equine Scrotum Strap	S
165321	KRUUSE Equine Scrotum Strap	M
165322	KRUUSE Equine Scrotum Strap	L

KRUUSE Foal Frontal Bandage

KRUUSE Foal Frontal Bandage is a combination of a 3-dimensional elastic material with large adjustable areas of foam making this bandage indispensable in covering wounds and cuts in places difficult to bandage. Standard length of foreleg sleeves 22 cm / 9" 70-89 cm / 28"-35".

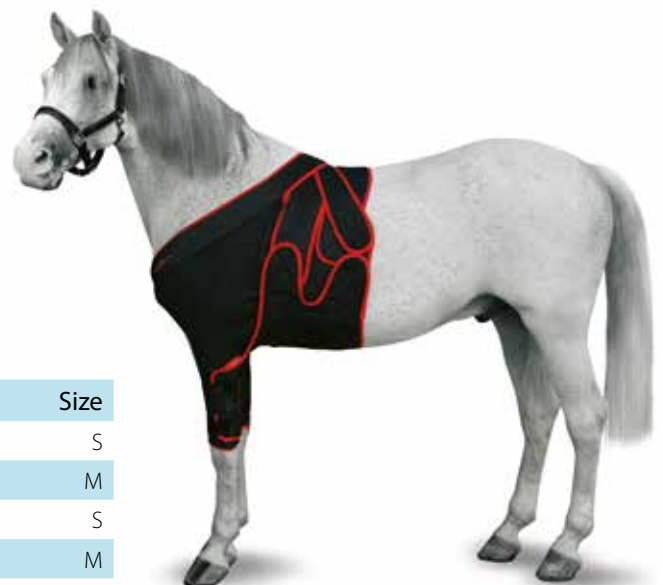

Scan QR code for more information

Cat. No	Description	Size
165212	KRUUSE Foal Frontal Bandage	S
165213	KRUUSE Foal Frontal Bandage	M
165214	KRUUSE Equine Frontal Bandage	S
165215	KRUUSE Equine Frontal Bandage	M
165216	KRUUSE Equine Frontal Bandage	L

KRUUSE Foal Face Mask

KRUUSE Foal Face Mask can be used as a post sinus surgery mask or as base for different eye masks for maximum protection. The mask is held in place with adhesive elastic straps, adjustable to size and contours. When using the face mask as a base for eye masks, it is important to choose the same size of face mask and eye protector. The fabric is breathable and machine-washable. The eye mask comes with one eye protector.



Cat. No	Description	Size
165240	KRUUSE Foal Face Mask	S
165241	KRUUSE Foal Face Mask	M

Scan QR code
for more
information



KRUUSE Foal Spare Full Eye Protector

The semi-flexible full eye protector made of 4 mm low-density polyethylene can be adjusted without effort to the right or left on the mask, without removing the entire mask.



Cat. No	Description	Size
165307	KRUUSE Foal Spare, Full Eye Protector (165223)	M

KRUUSE Foal Spare Half Eye Protector

The semi-flexible half eye protector made of 4 mm low density polyethylene can be adjusted independently at the right or left on the mask.



Cat. No	Description	Size
165311	KRUUSE Foal Spare, Half Eye Protector (165233)	M

KRUUSE Equine Face Mask

A full eye protector mask to be used as a post sinus surgery mask or as base for different eye masks for maximum protection. The mask is held in place with adhesive elastic straps, adjustable to height and contours to avoid window edemas.

The semi-flexible full eye protector made of 4 mm low-density polyethylene can be adjusted without effort, at the right or left on the mask, without the need to remove the whole mask. The eye mask comes with one eye protector.



Cat. No	Description	Size
165242	KRUUSE Equine Face Mask	S
165243	KRUUSE Equine Face Mask	M
165244	KRUUSE Equine Face Mask	L
165338	KRUUSE Equine Eye Patch (165242)	S
165339	KRUUSE Equine Eye Patch (165243)	M
165340	KRUUSE Equine Eye Patch (165244)	L
165329	KRUUSE Equine Spare Full Eye Protector (165242)	S
165330	KRUUSE Equine Spare Full Eye Protector (165243)	M
165331	KRUUSE Equine Spare Full Eye Protector (165244)	L
165338	KRUUSE Equine Eye Patch (165242)	S
165339	KRUUSE Equine Eye Patch (165243)	M
165340	KRUUSE Equine Eye Patch (165244)	L



EQUIVET Eye Protector Kit

EQUIVET Eye Protector features the exclusive equine "Eye Patch". It effectively closes the horse's eye for maximum pain relief while allowing for constant monitoring of the injury or condition. The eye protector ensures rest to the eye, eliminating the need for surgical closure.

Each kit contains:


- Blinker hood with right or left ball cup
- 3 soft snap-on rings
- 2 "tear-pads" to absorb excess moisture
- 15 gauze pads, sterilised (4" x 4" or 3" x 3")
- Protective ventilated ball cup



Features and benefits:

- Prevents the horse from rubbing its eye
- Designed to help the horse keep its eye closed without suturing
- The mask fits the horse snugly and is highly protective to the injured eye, yet comfortable
- Keeps flies and insects away from the injured eye and provides ventilation during the healing process
- Provides unmatched level of hygiene
- Easy to use, also by horse owners

Cat. No	Description	Type	Size
230780	EQUIVET Eye Protector Kit	Left eye	S
230781	EQUIVET Eye Protector Kit	Right eye	S
230782	EQUIVET Eye Protector Kit	Left eye	M
230783	EQUIVET Eye Protector Kit	Right eye	M
230784	EQUIVET Eye Protector Kit	Left eye	L
230785	EQUIVET Eye Protector Kit	Right eye	L
230786	EQUIVET Eye Protector Kit	Left eye	XL
230787	EQUIVET Eye Protector Kit	Right eye	XL

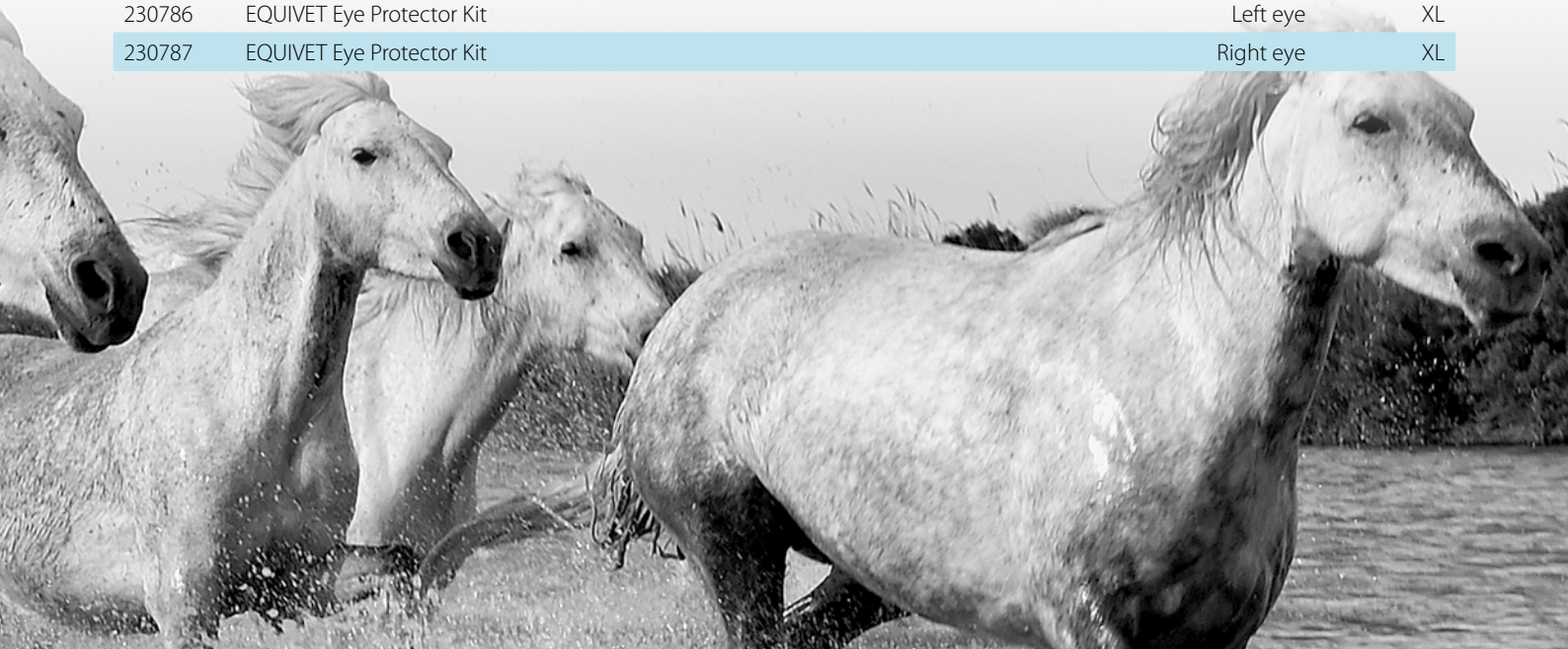

KRUUSE Equine Wound Chart

Dry 

Primary  Secondary  Tertiary 




- ▶ **Acute/traumatic wound**
- Evaluate the wound
- Clean the wound using KRUUSE wound lavage system
- Dress and bandage according to evaluation

Primary	Secondary	Tertiary



- ▶ **Necrotic**
- Debride the necrotic tissue and lavage
- Protect wound margins and surrounding skin from exudate

Primary	Secondary	Tertiary
KRUUSE Manuka G KRUUSE Manuka ND KRUUSE HydroGel KRUUSE Lite-Absorb	KRUUSE Foam KRUUSE Vet-Line KRUUSE Lite-Absorb	KRUUSE Vet-Plast KRUUSE Flex Bandages KRUUSE Soft-Flex KRUUSE Tubular Bandage

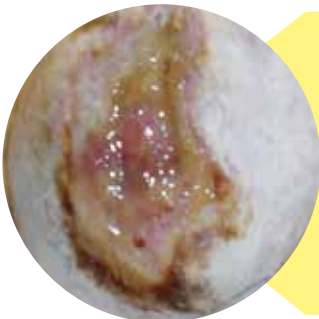

- ▶ **Sloughy**
- Debride devitalised tissue and lavage
- Protect wound margins and surrounding skin from exudate


Primary	Secondary	Tertiary
KRUUSE Manuka G KRUUSE Manuka ND KRUUSE HydroGel KRUUSE Lite-Absorb	KRUUSE Foam KRUUSE Vet-Line KRUUSE Lite-Absorb	KRUUSE Vet-Plast KRUUSE Flex Bandages KRUUSE Soft-Flex KRUUSE Tubular Bandage


Moderate exudate




Heavy exudate



Primary 

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
Secondary 

Tertiary 

Primary 

Secondary 

Tertiary 



KRUUSE Manuka G	KRUUSE Foam	KRUUSE Vet-Plast
KRUUSE Manuka AD	Steri-Protect	KRUUSE Flex Bandages
KRUUSE HydroGel	KRUUSE Vet-Line	KRUUSE Soft-Flex
KRUUSE Lite-Absorb	KRUUSE	KRUUSE Tubular Bandage
KRUUSE Super-Absorb	Lite-Absorb	
	KRUUSE Super-Absorb	

KRUUSE Manuka G	KRUUSE Foam	KRUUSE Vet-Plast
KRUUSE Manuka AD	Steri-Protect	KRUUSE Flex Bandages
KRUUSE Super-Absorb	KRUUSE Super-Absorb	KRUUSE Soft-Flex
		KRUUSE Tubular Bandage

KRUUSE Manuka G	KRUUSE Foam	KRUUSE Vet-Plast
KRUUSE Manuka AD	Steri-Protect	KRUUSE Flex Bandages
KRUUSE HydroGel	KRUUSE Vet-Line	KRUUSE Soft-Flex
KRUUSE Lite-Absorb	KRUUSE	KRUUSE Tubular Bandage
KRUUSE Super-Absorb	Lite-Absorb	
	KRUUSE Super-Absorb	

KRUUSE Manuka G	KRUUSE Foam	KRUUSE Vet-Plast
KRUUSE Manuka AD	Steri-Protect	KRUUSE Flex Bandages
KRUUSE Super-Absorb	KRUUSE Super-Absorb	KRUUSE Soft-Flex
		KRUUSE Tubular Bandage

