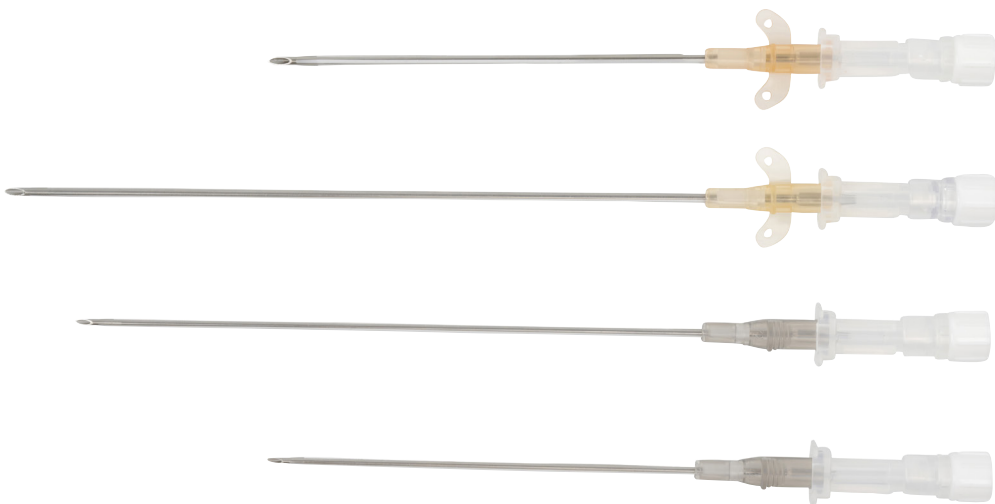
- GB KRUUSE Large Animal IV Catheters
- DK KRUUSE Large Animal IV Katetre
- SE KRUUSE IV-katetrar för stora djur
- DE KRUUSE IV-Katheter für Großtiere
- NL KRUUSE IV-katheters voor grote dieren
- ES Catéteres IV KRUUSE para animales grandes
- PT Cateteres IV KRUUSE Animais Grande Porte
- FR Cathéters IV KRUUSE pour animaux de grande taille
- IT Cateteri intravenosi per grandi animali KRUUSE
- PL Katetery dożylnie dla dużych zwierząt KRUUSE Large Animal IV
- FI KRUUSE Large Animal -IV-katetrit
- RU Внутривенные катетеры для крупных животных KRUUSE
- CN KRUUSE 大动物静脉留置针

Sterilized using ethylene oxide

Do not reuse

Do not re-sterilize

Does not contain latex

Does not contain DEHP

Consult instructions for use

Keep away from sunlight

Keep dry

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GB KRUUSE Large Animal IV Catheters

KRUUSE Large Animal IV Catheter assortment covers all needs and the catheters come in various types and sizes. They are available in both PU (polyurethane) and FEP (fluorinated ethylene propylene).

Instructions for use

The KRUUSE Large Animal IV Catheters are intended for venous catheterization in cases where continuing IV access is needed to ease procedures and avoid repeated venipuncture.

The Catheters can be used for administration of intravenous (IV) fluids, blood, plasma, IV drugs, and collection of blood from donor horses prior to transfusion.

Placement procedure guidelines

1. Clip the area over the jugular vein. Wipe with antiseptic gauze to remove superficial dirt.
2. Surgically scrub the area using gloves. Scrub in a circular motion, starting at the center and moving outward. Scrub four times, alternating with an aseptic product like chlorhexidine, then alcohol.
3. Subcutaneously inject lidocaine over the jugular vein.
4. Apply sterile gloves and repeat surgical scrub as described above.
5. Occlude jugular (or whatever vein used) vein by applying pressure in the jugular groove 2-3 inches below the catheterization site and visualize the raised vein.
6. Remove the plastic needle guard and introduce the catheter needle/stylet into the vein in approximately 35-45-degree angle.
7. Flash of blood into the needle hub confirms correct placement. Advance needle and catheter a few millimeters.
8. If there is no resistance and the blood continues to flow, hold the needle stylet in place and gently advance the catheter forward into the vein. Blood should flow freely from the catheter if in place.
9. Withdraw the needle, holding the catheter in place. Dispose of the needle into an approved sharps container, according to practice procedure.
10. Place the end cap on the catheter and secure in place by fixating the catheter to the skin by suturing around the hub or wings close to the insertion area. Skin glue can also be used.
11. Extension line, prefilled with heparinised saline, can be attached to the catheter, and secured in place to allow for multiple daily access. Flush the line.

NOTE


1. Always inspect the package to be sure it is intact, ensuring the catheter is sterile before use.
2. If resistance is felt by placement, the catheter may not be in the vein. Only attempt to reposition if the stylet is still in place.
3. Do not attempt to re-thread the catheter with the stylet once it is removed.
4. If long-term use is intended, PU catheters are least tissue irritating, non-thrombogenic and non-cracking.

Large Animal IV Catheter maintenance suggestions

Temporary IV catheters can remain in place for up to four days. The need of long-term IV access can be solved using a long-term catheter material such as PU, which allows an indwelling time for weeks -assuming strict aseptic technique, close monitoring, and maintenance. Inspect the catheter site ideally every 4 hours. Flush the catheter at least twice daily with heparinised saline. If IV Catheters are to remain in the vein for multiple days, flush every 4-6 hours. To ensure adequate hemostasis upon removal of the catheter, be sure to have a clean area and apply pressure with gauze for about two minutes.

Product details and re-order information

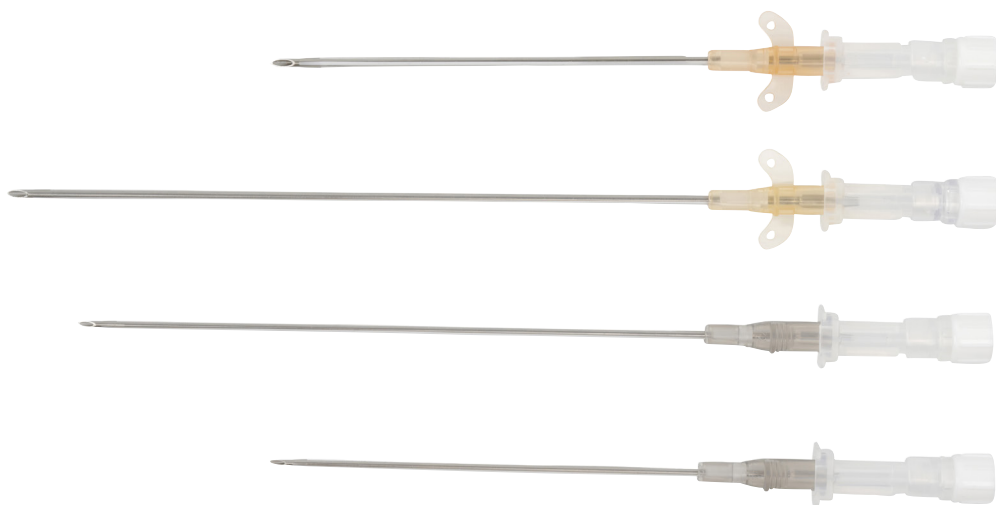Cat. No	Type	Material	Colour	Gauge	Length	Priming volume	Flow rate	I.D.	O.D.
122020	Pencil style	PU	● Orange	14 G	80 mm / 3.1"	0.30 ml	295 ml/min.	1.7 mm	2.1 mm
122021	Pencil style	PU	● Orange	14 G	120 mm / 4.7"	0.45 ml	270 ml/min.	1.7 mm	2.1 mm
122022	Pencil style	PU	● Grey	16 G	80 mm / 3.1"	0.25 ml	180 ml/min.	1.3 mm	1.7 mm
122023	Pencil style	PU	● Grey	16 G	120 mm / 4.7"	0.35 ml	170 ml/min.	1.3 mm	1.7 mm
122040	Pencil style	FEP	● Light blue	12 G	80 mm / 3.1"	0.45 ml	425 ml/min.	2.15 mm	2.7 mm
122041	Pencil style	FEP	● Light blue	12 G	120 mm / 4.7"	0.65 ml	410 ml/min.	2.15 mm	2.7 mm
122042	Pencil style	FEP	● Orange	14 G	80 mm / 3.1"	0.30 ml	295 ml/min.	1.7 mm	2.1 mm
122043	Pencil style	FEP	● Orange	14 G	120 mm / 4.7"	0.45 ml	270 ml/min.	1.7 mm	2.1 mm
122044	Pencil style	FEP	● Grey	16 G	80 mm / 3.1"	0.25 ml	190 ml/min.	1.3 mm	1.7 mm
122045	Pencil style	FEP	● Grey	16 G	120 mm / 4.7"	0.35 ml	170 ml/min.	1.3 mm	1.7 mm
122024	Wings	PU	● Orange	14 G	80 mm / 3.1"	0.30 ml	295 ml/min.	1.7 mm	2.1 mm
122025	Wings	PU	● Orange	14 G	120 mm / 4.7"	0.45 ml	270 ml/min.	1.7 mm	2.1 mm
122026	Wings	PU	● Grey	16 G	80 mm / 3.1"	0.25 ml	180 ml/min.	1.3 mm	1.7 mm
122027	Wings	PU	● Grey	16 G	120 mm / 4.7"	0.35 ml	175 ml/min.	1.3 mm	1.7 mm
122046	Wings	FEP	● Light blue	12 G	80 mm / 3.1"	0.45 ml	425 ml/min.	2.15 mm	2.7 mm
122047	Wings	FEP	● Light blue	12 G	120 mm / 4.7"	0.65 ml	410 ml/min.	2.15 mm	2.7 mm
122048	Wings	FEP	● Orange	14 G	80 mm / 3.1"	0.30 ml	295 ml/min.	1.7 mm	2.1 mm
122049	Wings	FEP	● Orange	14 G	120 mm / 4.7"	0.45 ml	270 ml/min.	1.7 mm	2.1 mm
122050	Wings	FEP	● Grey	16 G	80 mm / 3.1"	0.25 ml	180 ml/min.	1.3 mm	1.7 mm
122051	Wings	FEP	● Grey	16 G	120 mm / 4.7"	0.35 ml	170 ml/min.	1.3 mm	1.7 mm

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