

BUSTER Kennel Mats

Instructions for Use

The BUSTER Kennel Mats have been designed to fit standard kennel sizes. They are made of tough nylon that can be scrubbed and cleaned using all common veterinary disinfectants and parvocides. This helps to reduce the need for conventional washing and tumble drying of the bedding, hence reducing electricity usage. The BUSTER Kennel Mats eliminate the need to line kennels with newspaper and help to reduce clinical waste.

Inserting Gel Packs

Each BUSTER Kennel Mat comes with one or more gel packs depending on the size. Open the Velcro flap at the end of the mat. This will expose two zips. Unzip the zipper closest to the attachment of the flap. This will reveal a pocket running the length of the mat. Insert the activated and warmed/cooled gel pack(s) into the pocket. For the larger sizes of mat there are Velcro dividers in the main pocket that prevent the gel packs from rucking up. Once all the required gel packs have been inserted close the zipper and replace the Velcro flap.

Cleaning

Remove large amounts of debris or faecal material. The BUSTER Kennel Mat can be scrubbed with all the common veterinary disinfectants and parvocides. Once clean, rinse with clean water. Allow to drip dry. It is recommended that periodically the sealed foam pouch (accessed via the second zip under the Velcro flap) is removed and the BUSTER Kennel Mat cover is soaked over night in a disinfectant or parvocide. The cover should be rinsed thoroughly in clean water before being allowed to drip dry. Wipe the sealed foam pouch with disinfectant or parvocide. Re-insert the foam pouch into the cover.

DO NOT MACHINE WASH THE COVER AS THIS WILL AFFECT ITS WATERPROOF PROPERTIES

For veterinary use only



Cat. No 271720 - Small
 Cat. No 271721 - Medium
 Cat. No 271722 - Large
 Cat. No 271723 - X-Large