

Use of Oxygen

Each BUSTER ICU Cage comes with an oxygen accessories kit.

Contents

6 x Venturi – 1 of each colour green, red, yellow, white, blue and a blank (no holes)
1 x Nebuliser
1 x T piece
1 x Cap
1 x 2.1m oxygen tubing

Instructions for Use

Attach the blank Venturi (no holes) to one end of the oxygen tubing.
This then attaches to the oxygen supply.

For Nebulisation

Unscrew the top half of the nebuliser and add the chosen medication.
Keep the nebuliser vertically whilst screwing back together.
Attach the end of the oxygen tubing to the base of the nebuliser.
Attach the top of the nebuliser to the vertical arm of the T piece
Attach one of the horizontal arms of the T piece to the BUSTER ICU Cage.
Close the other horizontal arm with the cap provided.
Turn on the oxygen at sufficient flow rate to nebulise the chosen medication.

For Oxygen Therapy

Attach the appropriate coloured Venturi to the other end of the oxygen tubing.
Attach the Venturi to the vertical arm of the T piece.
Push one of the horizontal arms of the T piece into the port on the BUSTER ICU Cage.
Close the other arm of the T piece with the cap provided.
Turn on the oxygen to the appropriate flow rate.
The minimum oxygen flow rate is 2 litres per minute.

For veterinary use only

